Instructions for Use

TUBE CLAMPS

EN		Instructions for Use	3
CS		Návod k použití	8
SK		Indikácie použitia	13
RO		Indicații de utilizare	18
HR		Upute za uporabu	24
HU		Használati javallat	29
BG		Указания за употреба	36
SL		Indikácie pre použitie	42

ENGLISH

Medical Device

INTENDED USE

The device is intended as a connection component of a prosthetic system that replaces a missing lower limb.

Suitability of the device for the prosthesis and the patient must be evaluated by a healthcare professional.

The device must be fitted and adjusted by a healthcare professional.

For more detailed device information, see **Table 1**.

Indications For Use and Target Patient Population

- Lower limb loss, amputation, or deficiency
- No known contraindications

For weight limit and impact level, see **Table 2**.

GENERAL SAFETY INSTRUCTIONS

Warning: Use of a lower limb prosthetic device carries an inherent risk of falling which may lead to injury.

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

Warning: If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

Warning: Only use Össur replacement parts for the device for ensured durability.

The device is for single patient use.

ASSEMBLY INSTRUCTIONS

Warning: Ensure tube is fully inserted into Tube Clamp Adapter before tightening.

Warning: For secure fastening use medium strength threadlocker and apply rated torque twice, see **Table 1**.

Only applicable for female adapter:

Warning: When extreme alignment is necessary, use extended set screws to achieve secure fastening.

USAGE

Cleaning and care

Clean with a damp cloth and a mild soap. Dry with a cloth after cleaning.

Environmental Conditions

See which environmental conditions apply to the device in **Table 1**.

A Waterproof device can be used in a wet or humid environment and submerged in up to 3-meter-deep water for a maximum of 1 hour.

It can tolerate contact with: Salt water, chlorinated water, perspiration, urine, and mild soaps.

It can also tolerate occasional exposure to sand, dust, and dirt.

Continuous exposure is not allowed.

Dry with a cloth after contact with fresh water or humidity.

Clean with fresh water after exposure to other liquids, chemicals, sand, dust, or dirt and dry with a cloth.

A Weatherproof device can be used in a wet or humid environment and can tolerate being splashed by fresh water (e.g., rain), no submersion is allowed.

No contact with salt water or chlorinated water is allowed.
 Dry with a cloth after contact with fresh water or humidity.
 Clean with fresh water in case of accidental exposure to other liquids, chemicals, sand, dust, or dirt and dry with a cloth.

MAINTENANCE

The device and the overall prosthesis should be examined by a healthcare professional. Interval should be determined based on patient activity.

SPECIFICATIONS

Table 1 – Device Specifications

Part #	Description	Material	Tightening Torque			Environmental Conditions
			Pinch bolt	Mount screws	Set screws	
A-322100	4-Hole Tube Clamp AL	Aluminum	10 Nm	10 Nm	None	Weatherproof
A-333100	Male Tube Clamp AL	Aluminum & Stainless Steel	10 Nm	None	None	
A-342100	Female Pyramid Tube Clamp	Aluminum	10 Nm	None	15 Nm	
A-342430	Female Tube Clamp AL		10 Nm	None	15 Nm	
A-342410			10 Nm	None	15 Nm	
A-325100	4-Hole Tube Clamp TI	Titanium	10 Nm	10 Nm	None	Waterproof
A-335100	Male Tube Clamp TI		10 Nm	None	None	
A-345100	Female Tube Clamp TI		10 Nm	None	15 Nm	
AJ-345100	Female Tube Clamp Junior		5 Nm	None	8 Nm	
AJ-342100		Aluminum	5 Nm	None	8 Nm	Weatherproof

REPORT OF SERIOUS INCIDENT

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur prosthetic devices are designed and verified to be safe and compatible in combination with each other and custom-made prosthetic sockets with Össur adapters, and when used in accordance with their intended use.

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

Compliance

This device has been tested according to ISO 10328 standard to three million load cycles.

Depending on patient activity, this may correspond to 3-5 years of use.

Table 2 – Usage Specifications

Part #	Impact Level	Weight Limit	ISO 10328 - Label Text
A-322100	The device is for low to moderate impact use, e.g., walking.	100 kg	ISO 10328 - P5 -100 kg
A-333100			
A-342100			
A-342430			
A-342410	The device is for low to high impact use, e.g., walking and occasional running.	166 kg	ISO 10328 - P7 - 166 kg
A-325100			
A-335100			
A-345100	The device is intended for children and adolescent patients.	60 kg	ISO 10328 - P4 - 60 kg
AJ-342100		45 kg	ISO 10328 - P3 - 45 kg

ISO 10328 - “P” - “m”kg *)

*) Body mass limit not to be exceeded!

For specific conditions and limitations of use see manufacturer's written instructions on intended use!

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