


BIRD & CRONIN

Classic Wrist Brace

Application Instructions


1. Measure the wrist circumference to determine the proper size. Slide brace on the affected hand with the layer of stockinette on the back of the hand.

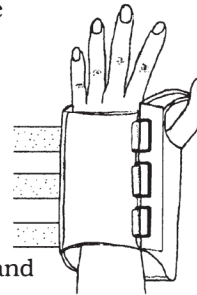

2. Thread hook and loop through D-rings securing the hook and loop closures. The metal stay should fit into the palm of the hand and extend down the forearm.

3. Trim the straps leaving approximately 1" - 2" of overlap.

NOTE: The metal stay has been positioned so the wrist will be set at 30° of extension. For patients with Carpal Tunnel Syndrome, the metal stay can be adjusted to 0° to 20° of extension.

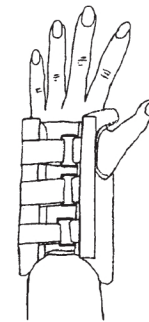

WASHING INSTRUCTIONS

Remove metal stay. Fasten hook and loop. Hand wash in warm water using a mild detergent. Air dry. Blot excess water with a towel, do not wring out.

Reorder No. Left	Reorder No. Right	Wrist Circumference
0814 5951	0814 5961	X-Small (up to 5 3/4")
0814 5952	0814 5962	Small (5 3/4" - 6 1/2")
0814 5953	0814 5963	Medium (6 3/4" - 7 1/2")
0814 5954	0814 5964	Large (7 3/4" - 8 1/2")
0814 5955	0814 5965	X-Large (8 3/4" +)

BIRD & CRONIN INC.

Designers and Manufacturers of Quality Medical Products

1200 Trapp Road • Eagan, Minnesota USA 55121

800/328-1095 • Fax 800/278-7934 • 651/683-1111 • Fax 651/688-9855

e-mail sales@birdcronin.com • www.birdcronin.com

©2000 Bird & Cronin, Inc. All Rights Reserved - 5951