
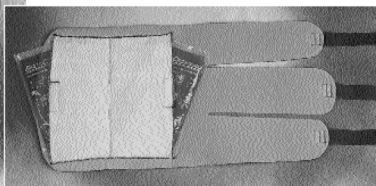


B-Cool™ Arthroscopic Knee Wrap - Universal

Application Instructions

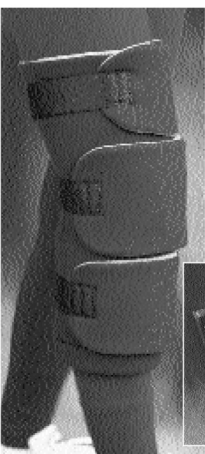

Reorder No.

0814 2640

0814 2640C

0814 2638

0814 2639

Size

Universal

Universal/ 6 per case

Universal with 2-Ice Packs

Universal with 4-Ice Packs

Length

14"

14"

14"

14"


CM

35.6cm

35.6cm

35.6cm

35.6cm

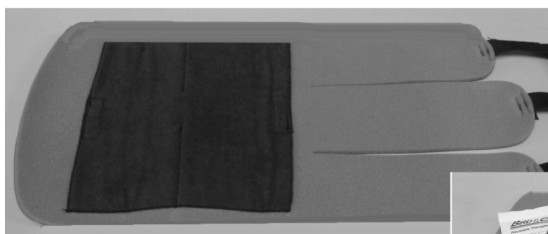

1. Open B-Cool™ Arthroscopic Knee Wrap - Universal.

2. Place ice packs in terry pockets.



3. Place the wrap by aligning the patella opening over the knee.



4. Secure the middle strap first and follow other two straps with hook and loop closure.



3. Adjust if necessary.

NOTE: This product should be used only under the supervision of a medical professional. It is not intended for unsupervised public use. If pain, swelling, changes in sensation or unusual reaction occur immediately contact your healthcare professional.

BIRD & CRONIN INC.

Designers and Manufacturers of Quality Medical Products

1200 Trapp Road • Eagan, Minnesota USA 55121

651/683-1111 • Fax 651/688-9855 • 800/328-1095 • Fax 800/279-7934

e-mail sales@birdcronin.com • www.birdcronin.com

©2009 Bird & Cronin, Inc. All Rights Reserved - 10000632CM