

Cold Therapy Lumbar Wrap

Application Instructions



1. Insert cold packs into pockets and secure closure.



2. Position lumbar wrap with cold packs positioned on affected area.

Refer to instructions on B-Cool Packs for proper use.



3. Secure wrap with hook closure .



4. Readjust if necessary.

Hand wash with mild detergent. Air dry.

Reorder No.
0814 3380

Size
Universal

Rib Measurement
Up to 40"

**B-Cool Ice Packs are sold separately*

Reorder No.
0814 0610

Size
6" x 10"(24/cs)

Unit
Each

BIRD & CRONIN INC.

Designers and Manufacturers of Quality Medical Products

1200 Trapp Road • Eagan, Minnesota USA 55121

651/683-1111 • Fax 651/688-9855 • 800/328-1095 • Fax 800/279-7934

e-mail sales@birdcronin.com

©2004 Bird & Cronin, Inc. All Rights Reserved - Form 3380