



BIRD & CRONIN

PREMIER WRIST BRACE

Application Instructions



1. Measure the wrist circumference to determine the proper size. Slide brace on the affected hand with the layer of stockinette on the back of the hand.
2. Thread hook and loop through D-rings securing the hook and loop closures. The metal stay should fit into the palm of the hand and extend down the forearm.
3. Trim the straps leaving approximately 1" - 2" (2.5-5.1cm) of overlap.

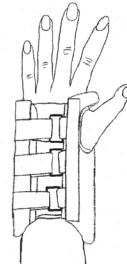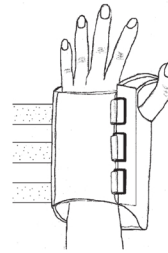

WASHING INSTRUCTIONS

Remove metal stay. Fasten hook and loop. Hand wash in warm water using a mild detergent. Air dry. Blot excess water with a towel, do not wring out.

Reorder No. Left	Reorder No. Right	Wrist Circumference
0814 4390	0814 4380	XX-Small (up to 5 3/4")(14.6cm)
0814 4391	0814 4381	X-Small (5 3/4" - 6 1/2")(14.6-16.5cm)
0814 4392	0814 4382	Small (6 1/2" - 7 1/4")(16.5cm-18.4cm)
0814 4393	0814 4383	Medium (7 1/4" - 8 1/4")(18.4-21.cm)
0814 4394	0814 4384	Large (8 1/4" - 9")(21.-22.9cm)
0814 4395	0814 4385	X-Large (9" - 10")(22.9-25.cm)

BIRD & CRONIN INC.

Designers and Manufacturers of Quality Medical Products
 1200 Trapp Road • Eagan, Minnesota USA 55121
 800/328-1095 • Fax 800/278-7934 • 651/683-1111 • Fax 651/688-9855
 e-mail sales@birdcronin.com • www.birdcronin.com
©2009 Bird & Cronin, Inc. All Rights Reserved -10004391CM