

SPRINT® AIR ANKLE WALKER



Application Instructions

1. Unfasten straps, remove anterior panel, open liner and remove bulb cover.
2. While seated, place leg in brace and position heel at the back of the brace. Wrap liner around the leg and foot and secure.
3. Replace the anterior panel.
4. Fasten the straps from the forefoot to the top. Alternate ankle strap placement allows for additional control to help prevent plantar flexion and also is beneficial for smaller ankles.
5. Adjust air bladder compression by pushing the bulbs on either side of the boot. To deflate the air bladders, open the cap of the smaller button next to the bulbs and push the button to release air.
6. Deflate the air bladders when removing the boot and follow the application instructions when the boot is put back on.

Adjustable Toe Guard

1. Toe Guard has 3 settings and can be adjusted to accommodate foot length.
2. To adjust Toe Guard pull guard upward, pull out guard on both sides from slots. Re-position guard into desired slot and pull downward to lock guard into place. Toe Guard may be removed if desired.

Reorder No.	Size	Female Shoe Size European Shoe Size	Male Shoe Size European Shoe Size	Unit
0814 1261	X-Small	Up to 5 35 - 36	Up to 4 38	Each
0814 1262	Small	5 - 8 37 - 39	4 - 7 39 - 41	Each
0814 1263	Medium	8 - 11 39 - 42	7 - 10 41 - 44	Each
0814 1264	Large	11 - 15 42 - 45	10 - 13 44 - 47	Each
0814 1265	X-Large	15+ 45+	13+ 47+	Each

MANUFACTURED FOR

BIRD & CRONIN INC.

Quality Medical Products To Help Improve Your Life

1200 Trapp Road • Eagan, Minnesota USA 55121

651/683-1111 • Fax 651/688-9855 • 800/328-1095 • Fax 800/279-7934

www.birdcronin.com • ©2014 Bird & Cronin, Inc. All Rights Reserved

10001261B