



U2 PEDIATRIC WRIST BRACE - UNIVERSAL

Application Instructions

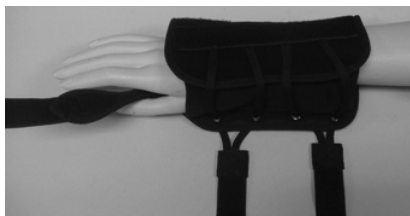

1. Open brace and position hand in wrist brace.

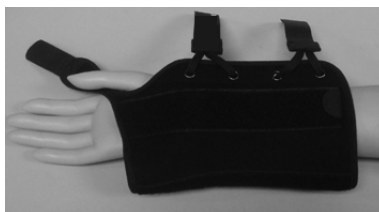

2. The palmar stay should line up below the middle crease of the hand.



3. Secure the thumb strap to the brace, aligning it between the thumb and index finger.



4. Wrap the lacing straps around wrist and attach to hook and loop. Re-adjust if necessary. The knuckles should be free for bending.

Reorder No.

0814 4022

0814 4023

Size

Universal Right

Universal Left

Unit

Each

Each

Hand Wash Cold - Air Dry

BIRD & CRONIN INC.

Quality Medical Products To Help Improve Your Life

1200 Trapp Road • Eagan, Minnesota USA 55121

651/683-1111 • Fax 651/688-9855 • 800/328-1095 • Fax 800/279-7934

e-mail sales@birdcronin.com

©2012 Bird & Cronin, Inc. All Rights Reserved 10004022