

# U2 Thumb Brace

## Application Instructions



1. Open brace and position hand in thumb brace.



2. The palmer stay should line up below the middle crease of the hand.



3. The thumb stay should follow the length of the thumb and arm.



4. Align the lacing dorsal stay with the middle of the knuckles.



5. Wrap the wide thumb strap around the thumb and secure to hook.



6. Secure the thinner strap between the thumb and index finger and attach to hook.



7. Wrap the lacing straps around wrist and attach to hook and loop. Re-adjust if necessary. The knuckles should be free for bending.

Hand Wash Cold - Air Dry

## ***BIRD & CRONIN INC.***

*Quality Medical Products To Help Improve Your Life*

1200 Trapp Road • Eagan, Minnesota USA 55121

651/683-1111 • Fax 651/688-9855

800/328-1095 • Fax 800/279-7934

e-mail [sales@birdcronin.com](mailto:sales@birdcronin.com) • ©2012 Bird & Cronin, Inc. All Rights Reserved  
10004030