




# Universal Wrist Lacer


## INSTRUCTIONS FOR USE

AW-CP020251 Rev D 10/15


This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for re-use on a second patient. This device is not intended to prevent injuries or reduce or eliminate the risk of injury.

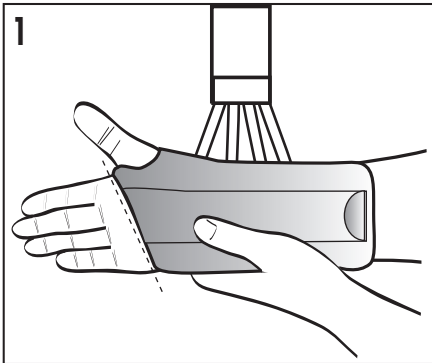


**1. Position the Brace**  
Place hand in the brace making sure the palm stay is below the crease in the palm of the hand and the palmer pad is in the center of the palm.

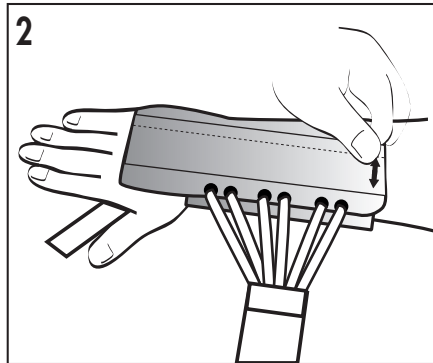


**2. Adjust Dorsal Stay**  
The dorsal stay should be centered on the mid-line of the arm. To adjust, remove the dorsal stay and re-secure in desired position.

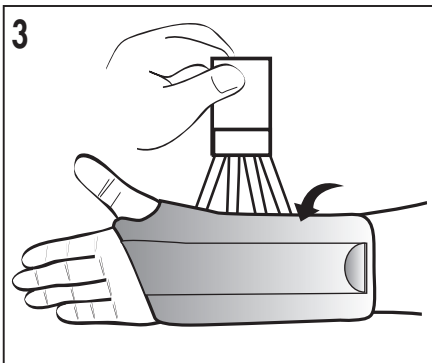

**3. Tighten Laces**  
While maintaining the brace position, tighten laces by pulling on the strap end. Wrap and secure strap.  
**NOTE: For 10" versions, pull and secure both strap ends.**

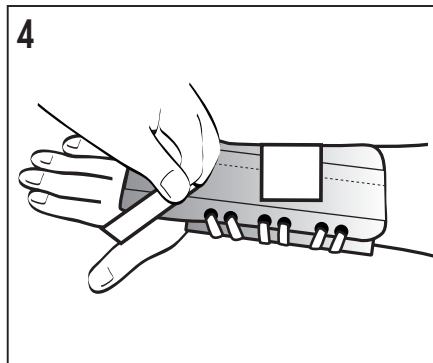

**4. Secure Remaining Strap**  
Wrap and secure the narrow strap.

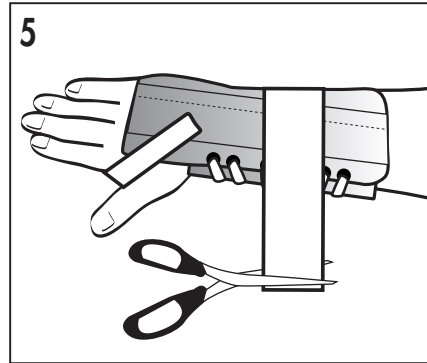

**5. Ensure Fit**  
Knuckles should not be obstructed, and brace should fit snug. Trim straps if necessary.

**INDICATIONS:** Indicated for immobilization and support for Carpal Tunnel Syndrome, rehab and post-cast support.

**CONTRAINDICATIONS:** Contraindicated for unstable fractures of the wrist.

**CAUTION:** Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention.


**CAUTION:** This device is not intended to prevent injury, or reduce or eliminate the risk of re-injury.

**CAUTION:** This device has been developed for the indications listed above. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.

**CAUTION:** Do not leave this device in a car on a hot day, or in any other place where the temperature may exceed 150°F (65°C) for any extended period of time.



© 2015 Breg, Inc. All Rights Reserved  
2885 Loker Ave E  
Carlsbad, CA 92010 USA  
Toll Free Tel: 1-800-321-0607  
Local: +1-760-795-5440  
Fax: +1-760-795-5295  
www.Breg.com

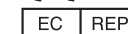

E/U authorized representative  
MDSS GmbH  
Schiffgraben 41  
D-30175 Hannover  
Germany