Explanation of Symbols

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Explication des symboles

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Verklaring van de symbolen

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as specified by hospital policies and procedures.

Application videos are available on www.deroyal.com.

SIZING

Universal

CLEANING AND/OR MAINTENANCE

To clean, hand wash with a mild fabric detergent in warm water. Do not bleach. Do not iron. Lay flat and open and allow to air dry. Inspect the product after washing to ensure it is intact. If the product is damaged, do not use it and replace it with a new one.

WARRANTY

DeRoyal products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

HEEL SUSPENSION BOOTS

**RX ONLY
SINGLE PATIENT USE
NON-STERILE
NOT MADE WITH NATURAL RUBBER LATEX**

IMPORTANT INFORMATION
Please read all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of the product.

Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

INTENDED USE

The DeRoyal Heel Suspension Boot is intended to be prescribed by a properly licensed practitioner to help prevent heel pressure ulcers, foot rotation and foot drop.

WARNINGS

- Do not allow patient to ambulate.
- Inspect product for proper fit immediately following patient application.
- Discontinue use of device and consult physician or properly licensed practitioner if device becomes damaged in any way, limiting its ability to perform as intended.
- Inspect product for damaged or missing components prior to patient application.

PRECAUTIONS

This product is to be fitted initially by a physician (or properly licensed practitioner) who is familiar with the purpose for which it is intended. The physician or practitioner is responsible for providing wearing instructions and precautions to other healthcare practitioners or healthcare providers involved in the patient's care as well as the patient. Consult your physician (or properly licensed practitioner) immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this product. Follow the instructions of your physician (or properly licensed practitioner) for length and duration of use.

Be sure to follow your facility's policies and guidelines for frequency of patient monitoring.

INSTRUCTIONS FOR USE

1. Release the hook tabs and loosen the straps.
2. Remove the foam block that is inside the boot.
3. Position the patient's foot inside the boot, with the heel suspended over the heel opening.
4. Feed the straps through the D-ring buckles and secure hook closures.
5. Visually check to ensure the straps do not restrict the patient's circulation.
6. To help prevent hip/foot rotation, remove the adhesive backing from the foam block insert and place on the medial or lateral side of the boot.
7. The patient's skin should be assessed frequently and the boot should be removed and placed back on the patient as specified by hospital policies and procedures. **NOTE:** Perform periodic skin assessment

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All Rights Reserved.