



PF STABILIZER WITH HINGE

INSTRUCTIONS FOR USE

Single Patient Use Only

BEFORE USING THIS DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE:

- Lateral patella subluxation
- Chronic patellar mal-tracking
- Chondromalacia

WARNINGS & PRECAUTIONS:

If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

CLEANING INSTRUCTIONS:

1. Clean material with water and a mild cleanser.
2. Lay flat and air dry ONLY.

WARRANTY:

Manufacturer guarantees all products to be free from defects in materials and workmanship for a period of 6 months from date of purchase.

CAUTION:

Federal Law (U.S.A.) restricts this device to sale by or on the order of a licensed Health Care Professional.

PART NUMBER: 300081-XX; 300082-XX; 300091-XX, 300092-XX.

-XX	Size	Circumference 6" above mid patella
-01	XS	13.00" - 15.50"
-03	SM	15.50" - 18.00"
-05	MD	18.00" - 20.50"
-07	LG	20.50" - 23.00"
-08	XL	23.00" - 26.00"
-09	2XL	26.00" - 29.00"

EUROPEAN REPRESENTATIVE:



EMERGO EUROPE
Molenstraat 15
2513 BH, The Hague
The Netherlands
Phone: +31.70.345.8570
Fax: +31.70.346.7299

UNITED SURGICAL
2235 Pennsylvania St.
Fort Wayne, IN 46835
USA

APPLICATION INSTRUCTIONS

1. Loosen all of the straps.



2. While sitting or standing, slide your foot through the top of the brace and pull the brace up the leg until the patella is centered in the patella cutout.



3. Tension the strap below the patella and fasten securely. Repeat with the strap above the patella.



4. Tension the "flap" below the patella and fasten securely. Repeat with the "flap" above the patella.




5. Walk around. Adjust the strap tension as needed. Ensure that the patella is centered.




Adjusting the Strap Length

The length of the straps may be adjusted if they are too long.

6. Remove the "alligator tab" from the end of the strap by peeling the Velcro.

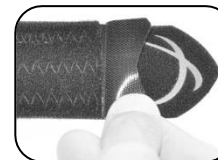

7. Using scissors, cut the strap to the desired length and reattach the "alligator tab" to the end of the strap. Apply brace as described above.

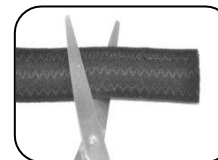

United Ortho Enterprises

2235 Pennsylvania St. Fort Wayne, IN 46803
Phone: 800-227-8748; Fax: 877-688-7975
www.United-Ortho.com