



FORM FIT® NEOPRENE KNEE SLEEVE



Life Without Limitations

Form Fit® Neoprene Knee Sleeve

1/8" - Open Patella

Part#	Size	Fits Circ.	U/M
302013	Small	11.5" - 13"	1 ea
302015	Medium	13" - 14.5"	1 ea
302017	Large	14.5" - 16"	1 ea

1/8" - Closed Patella

Part#	Size	Fits Circ.	U/M
312013	Small	11.5" - 13"	1 ea
312015	Medium	13" - 14.5"	1 ea

1/8" - Donut Buttress

Part#	Size	Fits Circ.	U/M
303013	Small	11.5" - 13"	1 ea
303015	Medium	13" - 14.5"	1 ea
303017	Large	14.5" - 16"	1 ea

1/8" - U Shaped Buttress

Part#	Size	Fits Circ.	U/M
304013	Small	11.5" - 13"	1 ea
304015	Medium	13" - 14.5"	1 ea
304017	Large	14.5" - 16"	1 ea

1/4" - Open Patella

Part#	Size	Fits Circ.	U/M
402013	Small	11.5" - 13"	1 ea
402015	Medium	13" - 14.5"	1 ea
402017	Large	14.5" - 16"	1 ea
402018	XLarge	16" - 17.5"	1 ea
402019	XXLarge	17.5" - 19"	1 ea

1/4" - Closed Patella

Part#	Size	Fits Circ.	U/M
412013	Small	11.5" - 13"	1 ea
412015	Medium	13" - 14.5"	1 ea
412017	Large	14.5" - 16"	1 ea
412018	XLarge	16" - 17.5"	1 ea
412019	XXLarge	17.5" - 19"	1 ea


Available in Blue or Black. Add BLU for blue or BLK for Black to part# when ordering.
Measure circumference at mid-patella.



1



2

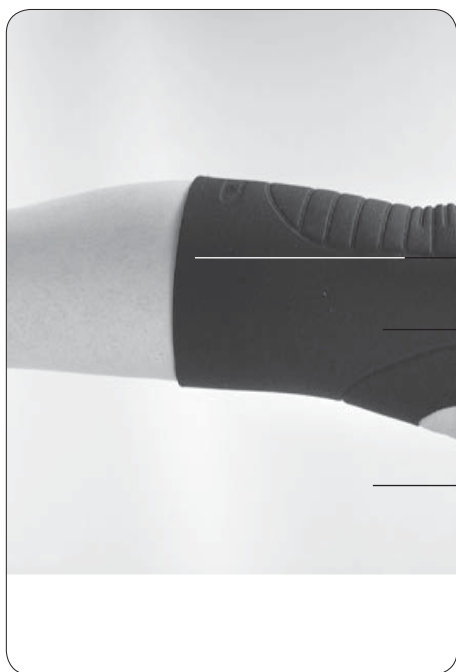

A.

B.

C.

Form Fit® Neoprene Knee Sleeve

USES

Post-operation rehabilitation, tendonitis, bursitis, and arthritis, sprains and strains.

CAUTION

Patients should be instructed to inspect their knee and the product before and after each use.

Modifications to this product are the sole responsibility of the prescribing physician.

PHOTO CAPTIONS

- A. Unique accordion design
- B. Open or closed patella
- C. Knee sleeve

FITTING INSTRUCTIONS

1. Place foot through larger opening first and slide brace onto leg.
2. Position patella opening over patella. If closed patella use marks or lines as guide.

WASHING INSTRUCTIONS

1. The brace can be cleaned using a mild detergent and cold water. Do not squeeze or wring out the brace. Do not wash in washing machine.
2. Lay brace out flat on a towel and air dry. Do not place in dryer. Do not place near heater.

Form Fit® Neoprene Knee Sleeve is covered by one or more of the following Patent No. 5,334,135; 5,695,452. Other U.S. and foreign patents granted and pending. ® indicates registration in U.S.A. and selected countries.

Ossur North America

27412 Aliso Viejo Parkway
Aliso Viejo, CA 92656, USA
Tel: +1 800 233 6263
Tel: +1 949 362 3883
Fax: +1 949 362 3888
ossurusa@ossur.com

Ossur - Braces and Support

742 Pancho Road
Camarillo, CA 93012, USA
Tel +1 805 484 2600
Fax +1 800 284 4356

Ossur - Custom Braces

19762 Pauling
Foothill Ranch, CA 92610, USA
Tel +1 949 859 4407
Fax +1 800 453 4567

Ossur Spinal

Mid Atlantic Corporate Center
1414 Metropolitan Ave.
P.O. Box 566
Thorofare, New Jersey 08086, USA
Tel: +1 856 848 0525
Fax: +1 856 845 1148
spine@ossur.com

Ossur Canada

12111 Jacobson Way
Richmond, BC
V6W 1L5, Canada
Tel: +1 604 241 8152
Fax: +1 604 241 8153

Ossur Europe

P.O. Box 120
5690 AC Son en Breugel
The Netherlands
Tel: 00800 3539 3668
Tel: +31 499 462840
Fax: +31 499 462841
saleseurope@ossur.com

Ossur Europe

Kundenservice Deutschland
Römerfeldstraße 2
50259 Pulheim, Germany
Tel: +49 (0) 2238 30 58 50
Fax: +49 (0) 2238 30 58 01
info-deutschland@ossur.com

Ossur Nordic

Salagatan 16a
P.O. Box 67
751 03 Uppsala, Sweden
Tel: +46 1818 2200
Fax: +46 1818 2218
info@ossur.com

Ossur International Markets

Grjóthals 5
110 Reykjavík, Iceland
Tel: +354 515 1300
Fax: +354 515 1370
international@ossur.com

Ossur Asia Pacific

2 Redbank Road
Northmead NSW 2152
Australia
Tel +61-2-9630-9206
Fax +61-2-9630-9268
info-asiapacific@ossur.com

Ossur Head Office

Grjóthals 5
110 Reykjavík, Iceland
Tel + 354 515 1300
Fax +354 515 1366
mail@ossur.com

