




Instructions for Use

BALANCE™ FOOT J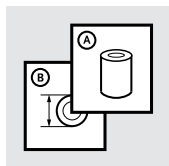


		3
EN	Instructions for Use	5
DE	Gebrauchsanweisung	6
FR	Notice d'utilisation	7
ES	Instrucciones para el uso	8
IT	Istruzioni per l'uso	9
NO	Bruksanvisning	10
DA	Brugsanvisning	11
SV	Bruksanvisning	12
EL	Οδηγίες χρήσης	13
FI	Käyttöohjeet	14
NL	Gebruiksaanwijzing	15
PT	Instruções de Utilização	16
PL	Instrukcja użytkowania	17
CS	Návod k použití	18
TR	Kullanım Talimatları	19
RU	Инструкция по использованию	20
JA	取扱説明書	21
ZH	中文说明书	22
KO	사용 설명서	23

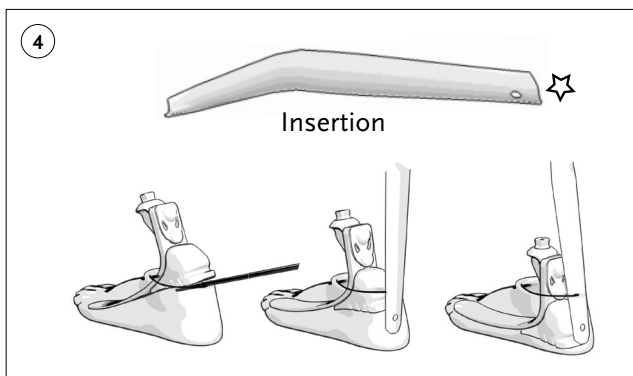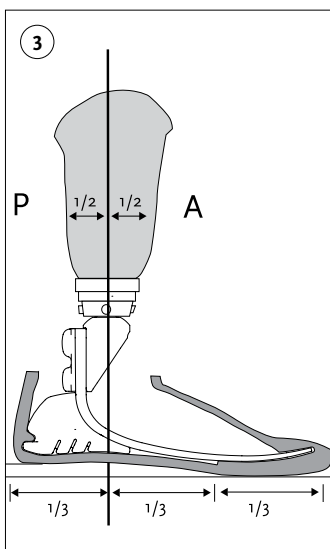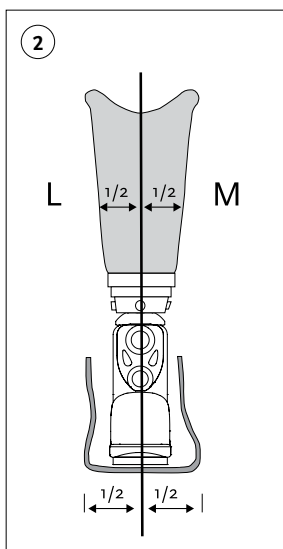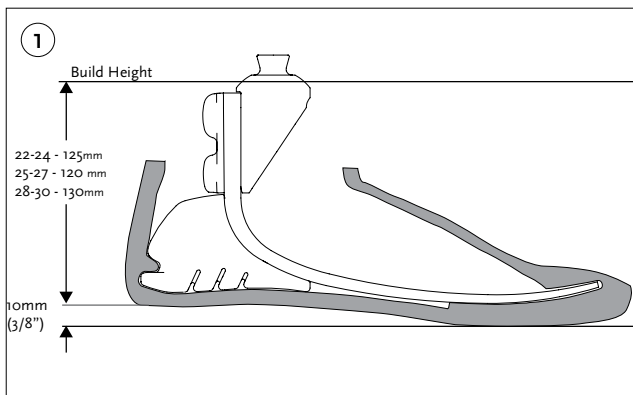5

Removal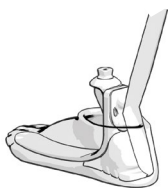

ENGLISH

TECHNICAL SPECIFICATIONS (Figure 1)

- Male pyramid connection
- Clearance: 125mm (22-24)
120mm (25-27)
130mm (28-30)
- Net weight: Size 27, cat2 w/ pyramid: 322g
- Foot cover size 27: 254g
- MAX 136kg (300lbs).
- Available in sizes 22-30cm.

INDICATIONS FOR USE

- Balance Foot J is rated for low impact levels (community ambulators).
- Not suitable for sport and high-impact activities.

ALIGNMENT: Figure 2

BENCH ALIGNMENT (Figure 2 & 3)

1. Fit the foot cover (**Figure 3 & 4**).
2. Adjust to appropriate heel height (using the shoe).
3. Introduce appropriate socket angles.
4. The load line should fall through the 1/3rd mark on the brim of the foot cover. The mark represents the junction of the posterior and the middle third portion of the foot.

Dynamic Alignment

The foam heel provides cushioning at heel strike. The toe provides forward progression at toe off.

The heel to toe action can be influenced by:

- A-P positioning of foot
- Dorsi-Plantarflexion

SHOEHORN

When removing and installing the cover use shoehorn to avoid damage to foot or cover (**Figure 4 & 5**).

CAUTION

Be sure to dry the foot inside and out after using it in a wet environment.

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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