




Instructions for Use

---

CHEETAH<sup>®</sup> XTREME &  
CHEETAH<sup>®</sup> XTEND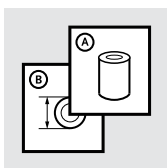

|    |                           |    |
|----|---------------------------|----|
|    |                           | 4  |
| EN | INSTRUCTIONS FOR USE      | 6  |
| DE | GEBRAUCHSANWEISUNG        | 8  |
| FR | NOTICE D'UTILISATION      | 10 |
| ES | INSTRUCCIONES PARA EL USO | 12 |
| IT | ISTRUZIONI PER L'USO      | 14 |
| DA | BRUGSANVISNING            | 16 |
| SV | BRUKSANVISNING            | 18 |
| NL | GEBRUIKSAANWIJZING        | 20 |
| PT | INSTRUÇÕES DE UTILIZAÇÃO  | 22 |
| JA | 取扱説明書                     | 24 |

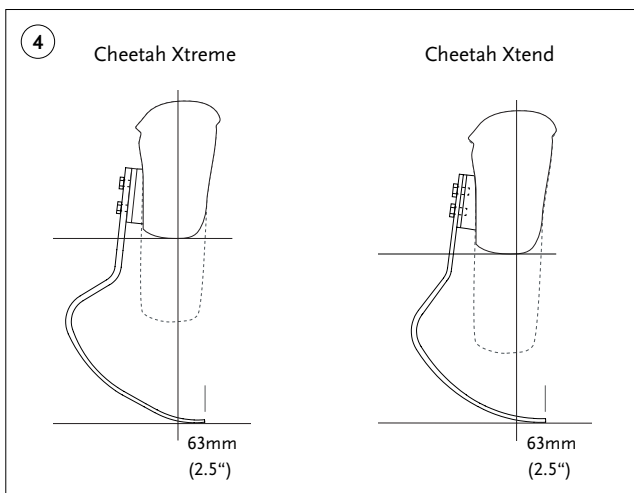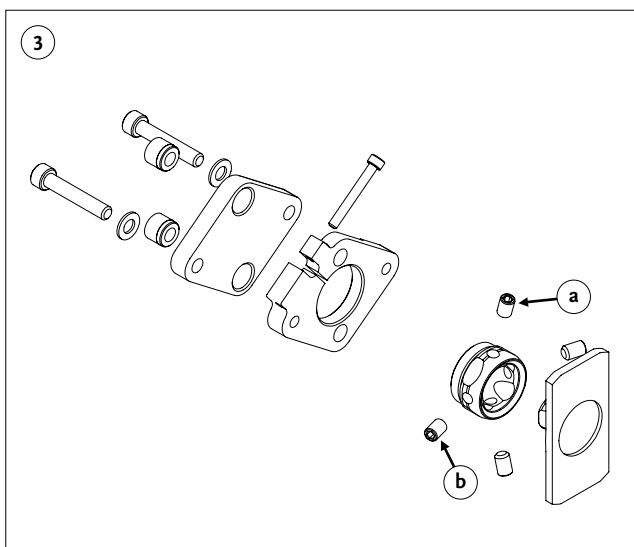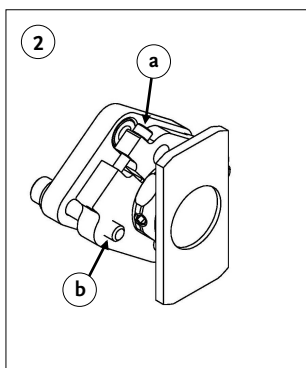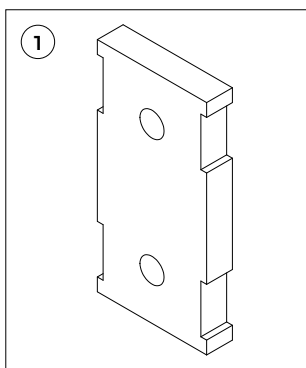5

6

# ENGLISH

## CHEETAH® XTREME & CHEETAH® XTEND

### Cheetah System

- Custom foot model for sprinting and recreational running.
- Maximum build height
  - Cheetah Xtreme 530mm (20 14/16")
  - Cheetah Xtend 451mm (17 3/4")
- Minimum build height
  - Cheetah Xtreme 375mm (14 3/4")
  - Cheetah Xtend 330mm (13")
- Maximum weight 147 kg (325lbs)
- One foot size

## LAMINATION CONNECTOR (Figure 1)

### FSX50009 (1-9)

- Lamination connector is laminated directly into the posterior socket wall.
- Placement of the lamination connector is recommended in the middle 1/3 of the socket for easier height adjustment.
- Make sure the socket surface is flat over the lamination connector to ensure good load distribution on the foot module when connected.

Bolts go through blade into lamination connector in socket. Apply Loctite 410 to all threads and torque to following specifications:

| Category | Bolt Size | Torque              |
|----------|-----------|---------------------|
| 1-9      | 3/8"      | 45 Nm ( 33 ft-lbs ) |

The Cheetah® Xtreme and Cheetah® Xtend may also be connected by direct lamination.

## FRICTION PADS

Always use friction pads between foot module and socket.

## ALIGNMENT ADAPTER FSX50007 (Figure 2&3)

**IMPORTANT:** Only use alignment adapter for dynamic alignment adjustments.

- Determine adapter to socket position according to athlete height requirements.
- Attach alignment adapter to check socket using epoxy or bonding compound.
- Reinforce attachment using Tech Form Cast tape.
- Use adjustment screws to adjust foot contact point following alignment recommendations:

### – Figure 2

- a. Inversion/eversion foot adjustment
- b. Height adjustment

### – Figure 3

- a. Dorsi/plantar flexion adjustment screw
- b. Internal/external rotation adjustment screw

## **BENCH ALIGNMENT**

### **Sagittal plane (Figure 4)**

- Determine vertical socket position in alignment jig.
- Load line bisects socket and falls through the ground contact point or approximately 63mm (2.5") posterior to end of toe.
- Clamp foot into a 7° flexion position.

### **Coronal plane**

- Introduce appropriate socket abduction/adduction using a jig.
- Determine appropriate internal or external socket rotation.
- Outset the foot module approximately 12 mm (1/2")
- Achieve proper height approximately 38mm (1 1/2") taller than the sound side to compensate for vertical compression.

**NOTE:** The recommendations for bench alignment are used as a starting point. Definitive alignment will be achieved during dynamic assessment using the alignment adapter.

## **CUTTING FOOT MODULE**

- Ensure athlete has enough clearance during knee flexion.
- Trim foot module to the top end of the laminating connector.
- Cut foot module using a hack saw or band saw.

### **NIKE SPIKE PAD (Figure 5 & 6)**

- Sand the bottom surface of the foot module
- Bond the Nike Spike Pad using contact cement or industrial strength glue
- Follow glue manufacturer bonding instructions

## **MAINTENANCE**

The Cheetah Xtreme & Cheetah Xtend can be used in water but warranty terms only apply if the following conditions are met:

- The foot is cleaned with tap water after use in salt water and / or freshwater.
- Immediately following contact with water, take the the foot cover off the foot and dry thoroughly.
- The components used with the foot need to be waterproof as well.

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**EN – Caution:** Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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