Figure 8 Clavicle Splint

Life Without Limitations®

Instructions for Use

Össur Americas
27051 Towne Centre Drive
Foothill Ranch, CA 92610, USA
Tel: +1 (949) 382 3883
Tel: +1 800 233 6263
Fax: +1 800 831 3160
ossurusa@ossur.com

Össur Canada
120-11231 Dyke Road
Richmond, BC
V7A 0A1, Canada
Tel: +1 604 241 8152
Fax: +1 604 241 8153

Össur Europe
Ekkersrijt 4106-4114
P.O. Box 120
5690 AC Son en Breugel
The Netherlands
Tel: +800 3539 3668
Tel: +31 499 462840
Fax: +31 499 462841
info-europe@ossur.com

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ENGLISH

USES

Clavicle fractures and dislocations, selected soft tissue injuries and postural problems.

FITTING INSTRUCTIONS

1. Position the felt pad on back between the shoulder blades.
2. Wrap figure-8 arm straps around shoulders. Make certain the straps are not twisted and the arm pads are between the straps and the shoulders.
3. Feed the straps through the buckles and pull tight as desired. Press metal teeth into the strap and pull strap to fasten.
4. The arm pad should be adjusted so that it fits under the arm and in front of the shoulder. The straps should be tight enough to pull the shoulders back.

WASHING INSTRUCTIONS

This product may be cleaned using mild detergent and cold water. Do not bleach. Do not wash in washing machine or place in dryer. Lay brace flat on a towel to air dry. Do not squeeze.

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EN – Caution: Össur products and components are designed and tested to ISO 10328. Compatibility and compliance with this standard is achieved only when Össur products and components are used with other recommended Össur or authorized components. If un-usual movement or product wear is detected in a structural part of a prosthesis at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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