




Instructions for Use

FLEX-FOOT® ASSURE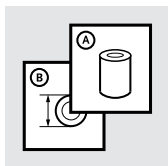

		3
EN	INSTRUCTIONS FOR USE	4
DE	GEBRAUCHSANWEISUNG	7
FR	NOTICE D'UTILISATION	10
ES	INSTRUCCIONES PARA EL USO	13
IT	ISTRUZIONI PER L'USO	16
NO	BRUKSANVISNING	19
DA	BRUGSANVISNING	22
SV	BRUKSANVISNING	25
EL	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ	28
FI	KÄYTTÖOHJEET	31
NL	GEBRUIKSAANWIJZING	34
PT	INSTRUÇÕES DE UTILIZAÇÃO	37
PL	INSTRUKCJA UŻYTKOWANIA	40
CS	NÁVOD K POUŽITÍ	43
TR	KULLANIM TALIMATLARI	46
RU	ИНСТРУКЦИЯ ПО ИСПОЛЬЗОВАНИЮ	48
JA	取扱説明書	51
ZH	中文说明书	54
KO	사용 설명서	57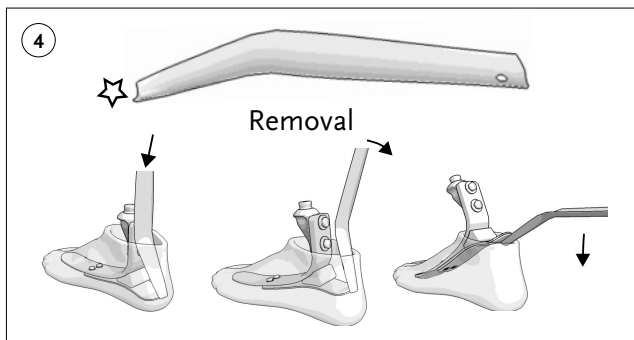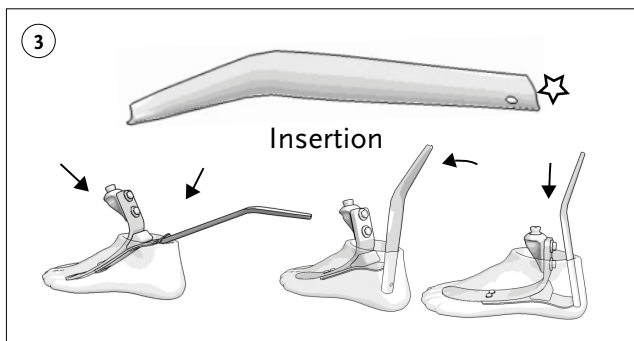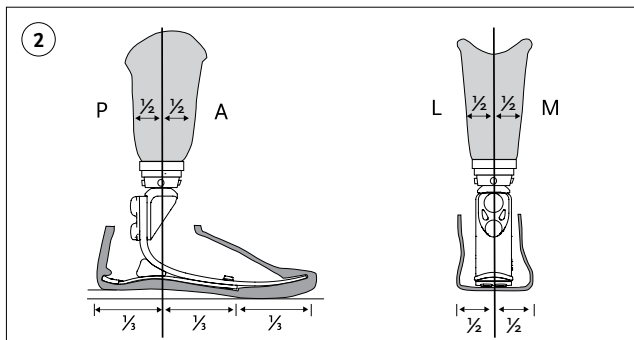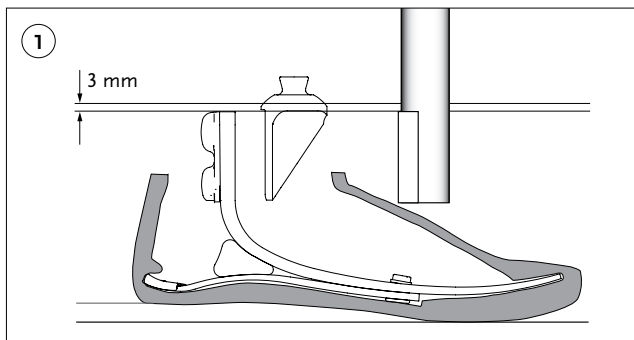

ENGLISH

Flex-Foot® Assure is to be used exclusively for lower limb amputees.

TECHNICAL SPECIFICATIONS: (Figure 1)

Tube option

- Sizes 19–24 MIN clearance 124 mm (4 7/8")
- Sizes 25–27 MIN clearance 119 mm (4 11/16")
- Sizes 28–30 MIN clearance 124 mm (4 7/8")

Pyramid option

- Sizes 19–24 MIN clearance 130mm (5 1/8")
- Sizes 25–27 MIN clearance 125mm (4 15/16")
- Sizes 28–30 MIN clearance 130mm (5 1/8")

Net weight

- Size 27, cat2 w/ pyramid: 332g (11.7oz)
- Foot cover size 27: 243g (8.6oz)

Weight limit:

- MIN 45kg (99lbs)
- MAX 136kg (300lbs)

Available in sizes 19–30cm

INDICATIONS FOR USE

- Flex-Foot Assure is rated for low impact levels (community ambulators).
- Not suitable for sport and high-impact activities.

ALIGNMENT: (Figure 2)

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- Provides smooth roll over and improved cushioning at heel strike.
- The heel wedge is an essential component for Flex-Foot Assure function.

! THE HEEL WEDGE SHOULD ALWAYS BE USED WITH THE FOOT.

BENCH ALIGNMENT

1. Fit the foot cover (**Figure 3 & 4**).
2. Adjust to appropriate heel height (using the shoe).
3. Introduce appropriate socket angles.
4. The load line should fall through the 1/3rd mark on the brim of the foot cover. The mark represents the junction of the posterior and the middle third portion of the foot.

DYNAMIC ALIGNMENT

- The heel stores energy at heel strike and slowly releases it to assist tibial progression.
- The forward momentum results in the toe being loaded for optimum energy release at toe off.
- The heel wedge greatly influences Flex-Foot Assure dynamic functions.

Heel too hard

Symptoms

- Rapid heel to toe movement.
- Poor control of the prosthesis at heel strike.
- Minimal energy return feeling.
- Knee may buckle.

Actions

- Shift socket posteriorly or foot anteriorly.
- Plantar-flex the foot.
- Review category selection.

Heel too soft

Symptoms

- Foot will come to flat position too early (sinking into a hole feeling).
- Extra energy is required to climb up over the toe.
- Toe will feel too stiff.
- Knee may hyperextend.

Actions

- Shift socket anteriorly or foot posteriorly.
- Dorsi-flex the foot.
- Review category selection.

HEEL ASSEMBLY

If it is necessary to replace or service the heel component use Loctite 410 and torque to following specifications:

Size	Bolt Size	Heel Bolt Torque
19–21	M6	14 Nm (10ft-lbs)
22–30	1/4"	14 Nm (10ft-lbs)

COMPONENT OPTIONS

30 mm fiberglass tube kits

Product No.	Cat.	Length	Weight
SFX00001	1–3	279 mm /11"	292 g /0.3 oz
SFX00002	1–3	381 mm /15"	333 g /11.7 oz
SFX00003	4–5	279 mm /11"	331 g /11.7 oz
SFX00004	4–5	381 mm /15"	292 g /10.3 oz

Male pyramid adapter (Metric) w/bolts

Product No.	Cat.	Weight
SFX00165	1–5	110 g /5.5 oz

Shoe horn

Product No.
CM100068
NOTE: For assembly instructions please refer to the applicable IFU 's.

WARRANTY

Össur provides a warranty of 36 months for the Flex-Foot Assure foot module and 2 months for the foot cover kit.

MAINTENANCE

The Flex-Foot Assure can be used in water but warranty terms only apply if the following conditions are met:

- The foot is connected to the rest of the prosthesis with a carbon tube.

- The foot is cleaned with tap water after use in salt water and / or freshwater.
- Immediately following contact with water, take the the foot cover off the foot and dry thoroughly.
- The components used with the foot need to be waterproof as well.

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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