Instructions for use

Universal Thumb Spica

Life Without Limitations®

.....	3
ⒺⒺ Instructions for use.....	5
ⒺⒺ Gebrauchsanweisung.....	6
ⒺⒺ Guide de fabrication.....	7
ⒺⒺ Instrucciones para el uso.....	8
ⒺⒺ Istruzioni per l'uso.....	9
ⒺⒺ Bruksanvisning.....	10
ⒺⒺ Brugsanvisning.....	11
ⒺⒺ Bruksanvisning.....	12
ⒺⒺ Käyttöohjeet.....	13
ⒺⒺ Gebruiksaanwijzing.....	14
ⒺⒺ Instruções de Utilização.....	15

EN - Caution: This product has been designed and tested based on single patient usage and is not recommended for multiple patient use. If any problems occur with the use of this product, immediately contact your medical professional.

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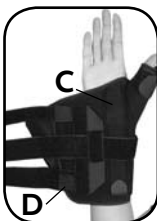
ENGLISH

USES

Gamekeeper's Thumb, DeQuervain's Tendonitis, selected scaphoid injuries, postcast healing, soft-tissue injuries.

CAUTION

Patients should be instructed to inspect their hand, wrist, thumb, and product before and after each use. Do not move strap pod after fitting.

PHOTO CAPTIONS

- A. Lycra tongue contact closure
- B. Softgood edge
- C. Palmer stay
- D. Strap pod
- E. Thumb hook tab

FITTING INSTRUCTIONS

1. Open brace to a flat position by detaching lycra tongue contact closure.

Slide injured thumb into brace.

3. Pull the lycra tongue contact closure over the top of wrist and attach to the loop tab on the inside lining of the brace.

3. Fasten the circumferential (middle) strap by inserting it through the middle of the three d-rings. Fold the strap back on itself around the entire wrist. Pull to a snug fit.

4. Fasten the remaining two straps by inserting them through the appropriate drings.

Fasten to the contact closure on the strap pod. Pull to a snug fit.

5. NOTE: This brace is pre-adjusted to fit a Medium size. If a larger or smaller size is desired, remove the brace and adjust the location of the strap pod accordingly. For smaller sizes, move it farther away from the softgood edge. For larger sizes, move it closer to the softgood edge.

6. To adjust size and fit of the thumb, detach and reposition the two thumb hook tabs as desired.

7. The strap pod straps may be trimmed as necessary after fitting

WASHING INSTRUCTIONS

This product may be cleaned using a mild detergent and cold water. Remove strap pod and aluminum stays (thumb and palmer stays). Do not bleach. Do not wash in washing machine or place in dryer. Lay brace out on a towel and air dry. Do not squeeze or wring out the brace.

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