









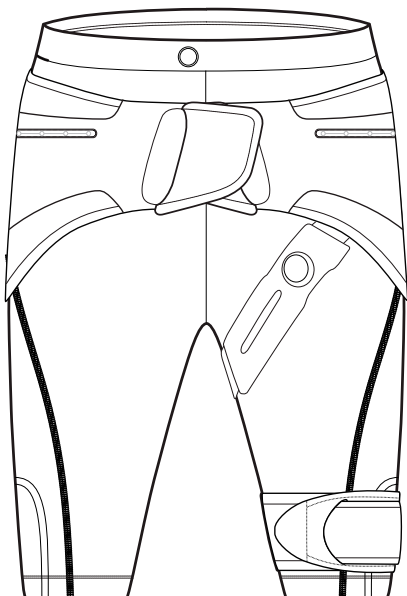
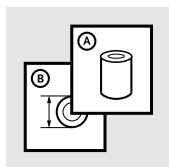
Instructions for Use

UNLOADER[®] HIP


		3
EN	Instructions for Use	6
DE	Gebrauchsanweisung	8
FR	Notice d'utilisation	10
ES	Instrucciones para el uso	12
NO	Bruksanvisning	14
DA	Brugsanvisning	16
SV	Bruksanvisning	18
FI	Käyttöohjeet	20
NL	Gebruiksaanwijzing	22
JA	取扱説明書	24

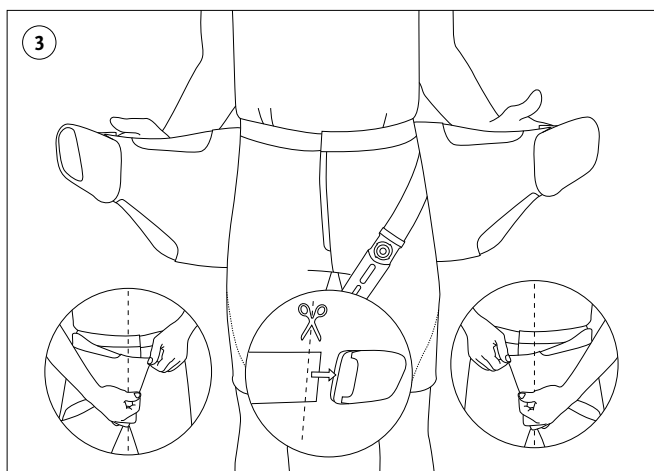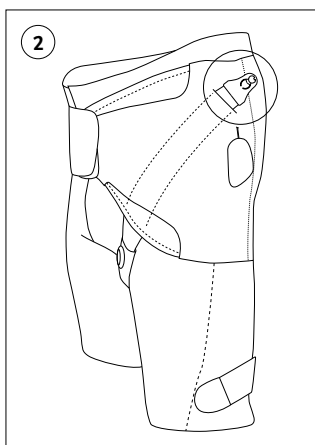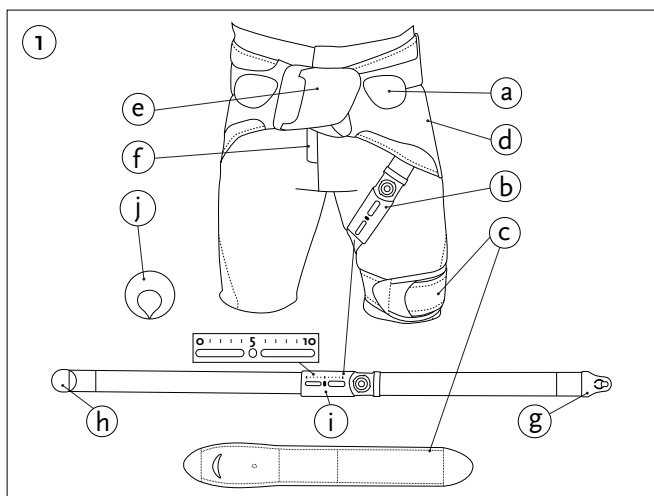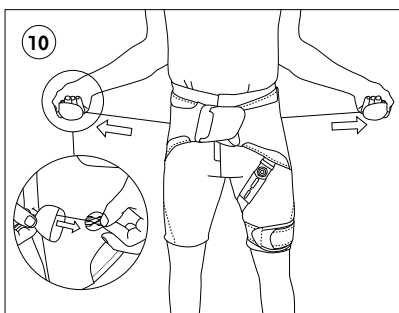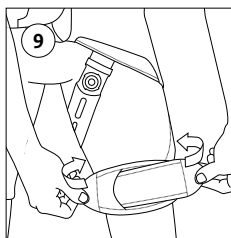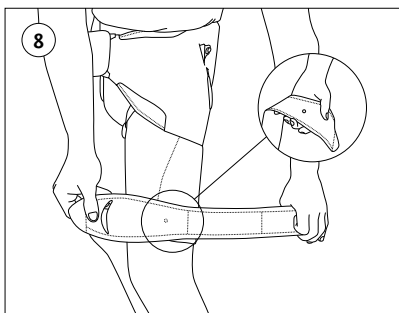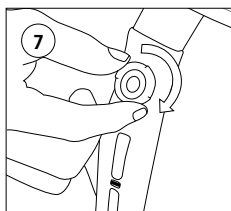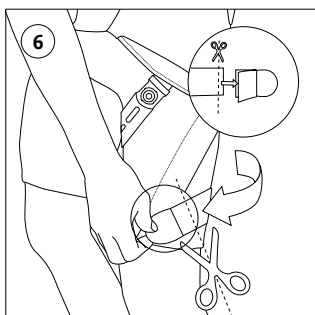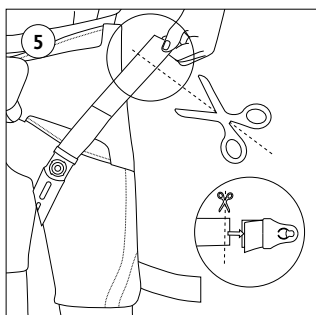
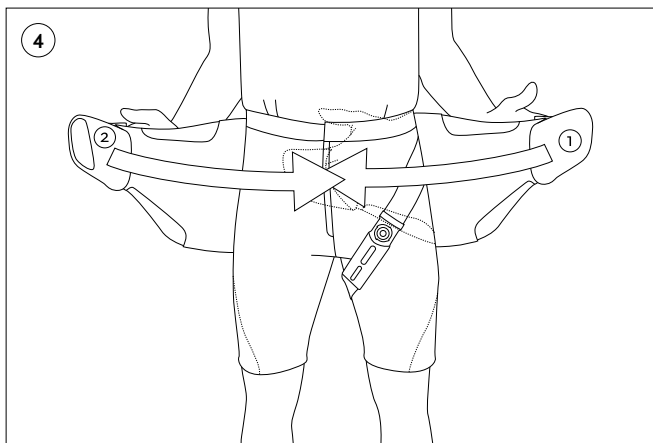
11

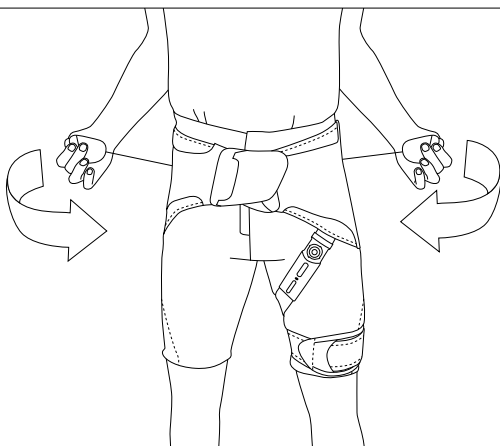12

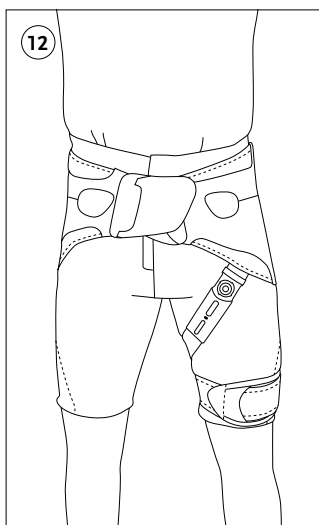13

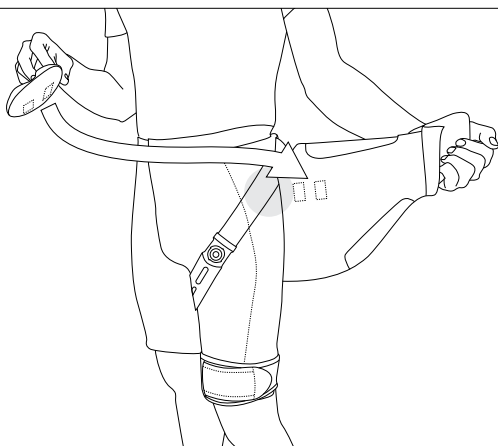INDICATIONS FOR USE

- Mild and moderate osteoarthritis of the hip.
- Conditions benefitting from repositioning of the hip and/or compression around the greater trochanters.

Note: For patients with peripheral vascular disease, neuropathy and sensitive skin, physician supervision is recommended. If redness appears, decrease usage of the brace and increase slowly as skin adapts to it.

PRODUCT OVERVIEW (FIGURE 1)

- a. Pulley System
- b. Rotation Control Strap
- c. Thigh Strap
- d. Belt Arm
- e. Front Closure
- f. Zipper
- g. Upper Alligator Clip
- h. Lower Alligator Clip
- i. SmartDosing Dial
- j. Trochanter Pad

FITTING INSTRUCTIONS

Note: At initial evaluation, fold belt arms and straps instead of cutting. When appropriate length has been determined, cut as needed to obtain final fit.

Rotation Control Strap Placement (Figure 2)

- Before donning the Unloader Hip, fasten Rotation Control Strap on the affected side by threading the Upper Alligator Clip through a slit in belt arm and attaching it to button hook
- SmartDosing Dial should be at ,0' mark
- Temporarily fasten lower strap end at the bottom of trousers so that it doesn't tear at the Lycra during the fitting
- Have user don the brace

Belt Arm Adjustment (Figure 3–4)

- Detach Front Closures from Belt Arms
- Fold/cut Belt Arms vertically so that they are aligned with mid-line
- Re-attach Front Closures
- Close Belt Arms so that they overlap tightly but comfortably around waist

Rotation Control Strap Adjustment (Figure 5–7)

- Detach Upper Alligator Clip and fold/cut strap so that the SmartDosing edge is positioned below Belt Arm line (approximately 2 cm / 0.79")
- Detach Lower Alligator Clip and fold/cut strap to obtain ideal tension, it should feel tight but not uncomfortable when wrapped around thigh and attached at outer side of leg
- With leg rotated slightly outwards, set SmartDosing dial at "5" mark – when rotating leg towards front and lifting it slightly, patient should experience agreeable tension in leg
- Repeat Lower Alligator Clip adjustment if necessary, objective is to

obtain adequate tension with SmartDosing at the “5” mark

Thigh Strap (Figure 8–9)

- Position Thigh Strap so that dot on thigh strap is positioned over dot on Rotation Control Strap. Thread slim end through opening and fasten securely

Pulley System Adjustment (Figure 10–12)

- Verify length of Pulley System strings by pulling at handles and attaching on Belt Arms
- If the handles overlap, shorten string by using the spooling system located inside handles
- Fasten Pulley System so that it sits tightly and comfortably on Belt Arms

Trochanter Pad (optional) (Figure 13)

Apply pad when peak pressure reduction on greater trochanter is needed and/or to increase sealing effect on labrum

- Position Trochanter Pad on the greater trochanter of the affected hip by attaching it to velcro under Belt Arm

FINAL CHECK

Verify that brace is sitting correctly on body, with all parts accurately positioned. Ask patient to walk around, sit down and stand up to ensure proper fit. Adjust as necessary, then proceed to cut straps and belt arms to ideal length

MAINTENANCE

- Hand wash using mild soap and rinse thoroughly or machine wash, using wash bag and a delicate program (no fabric softener)
- Avoid prolonged contact with salt and chlorinated water, rinse well in case it occurs
- Avoid prolonged exposure to strong sunlight
- Air dry (no tumble dry)

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