Rolyan® Original Hemi Arm Sling:

A653-LS, RS, LM, RM, LML, RML, LL, RL, LXL, and RXL

Rolyan® Hemi Arm Sling II:

A653-1 through A653-10

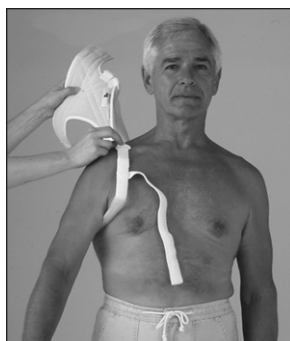
Rolyan® Hemi Arm Sling Cuff Adapter:

A653-CA and A653-11


Rolyan Custom Hemi Arm Slings

5598-64-01 through 5598-64-06 Cuffs

5598-64-07 through 5598-64-12 Straps


A

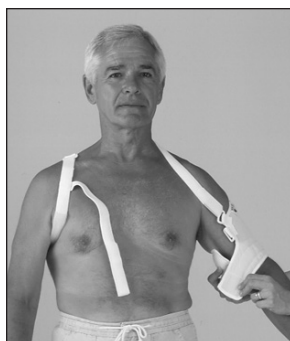B

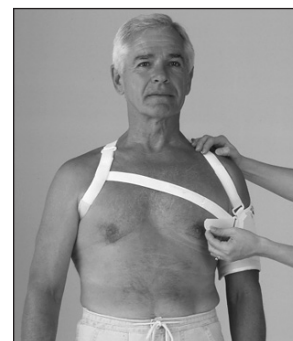C

D

Be sure to note which product you are using and follow the instructions that apply to that product. Unless otherwise stated, these instructions apply to all of the products listed.

INDICATIONS

- To support a subluxed shoulder

CONTRAINDICATIONS

- Not in the presence of a fracture

PRECAUTIONS

- **These products are to be fitted initially by a healthcare professional who is familiar with the purpose for which they are prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**
- **If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.**
- **Be careful that the sling and straps are not so tight that they interfere with circulation.**

CARE AND CLEANING

Rolyan Original Hemi Arm Sling

Rolyan Hemi Arm Sling Cuff Adapter

Fasten hook-and-loop closures. Hand wash with lukewarm water and mild soap. Wrap in towel to remove excess moisture and air dry. Be sure sling and cuff adapter are thoroughly dry before reapplication.

Rolyan Hemi Arm Sling II & Custom

Fasten hook-and-loop closures. Machine wash in warm water and machine dry. Be sure sling is thoroughly dry before reapplication.

APPLICATION

Instructions for one-handed application and removing sling are on center page—they can be given to patient.

Rolyan Original Hemi Arm Sling

Rolyan Hemi Arm Sling II, Rolyan Custom Hemi Arm Sling

1. Open hook-and-loop closures on arm cuff and chest strap (strap that is not attached to O-ring). Adjust the three remaining straps to their longest length.
2. With loose chest strap toward front of body and O-ring toward back, put unaffected arm through loop that has plastic strap protector and pull strap protector into underarm area (A).
3. Bring arm cuff across back and place over distal aspect of biceps (B). Secure hook-and-loop closure. The O-ring should not be centered on back—it should lie toward unaffected arm (C).
4. Adjust length of strap around unaffected shoulder so strap is snug.
5. Fasten chest strap (D).

6.

E

Adjust posterior strap and strap across clavicle to provide desired positioning of humerus.

7. After strap length and arm cuff circumference are determined, mark straps and cuff to indicate proper length for future application.
- #### Rolyan Hemi Arm Sling Cuff Adapter
8. With D-ring toward outside of arm, secure cuff adapter around forearm and wrist.
 9. Attach distal strap through thumb web space.
 10. Thread proximal strap through D-ring on cuff adapter and through D-ring on sling on unaffected shoulder (E).
 11. Adjust length of strap for desired elevation.

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Manufactured for: Patterson Medical

W68 N158 Evergreen Blvd.
Cedarburg, WI 53012, USA
Phone: 800-323-5547
Fax: 800-547-4333

Authorized European Representative Patterson Medical, Ltd.

Nunn Brook Road
Huthwaite, Sutton-in-Ashfield
Nottinghamshire, NG17 2HU, UK

United Kingdom

Phone: 08444 124 330
Fax: 08448 730 100
International
Phone: +44 (0) 1623 448 706
Fax: +44 (0) 1623 448 784

Revision 8; Effective 5/14
Patterson Medical Products, Inc.
9104.180C

One-handed application of Rolyan® Hemi Arm Slings—instructions for patient
Aanbrengen met één hand van de Rolyan® deelmitella voor arm - aanwijzingen voor de patiënt
Pour installer les écharpes de bras Rolyan® Hemi d'une seule main – consignes destinées au patient
Anlegen der Rolyan® Halbarmschlingen mit einer Hand – Anweisungen für den Patienten
Applicazione con una mano dei bendaggi a fionda per metà braccio Rolyan® - Istruzioni per il paziente
Aplicación con una mano de los cabestrillos para medio brazo Rolyan® —instrucciones para el paciente

ENGLISH

With hook-and-loop closure on chest strap open, secure three remaining straps and arm cuff at marked levels. Put arm cuff into position on affected arm. Make sure straps are untangled and hanging forward.

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A

ENGLISH

Put loop with plastic strap protector around unaffected arm and slide over shoulder.

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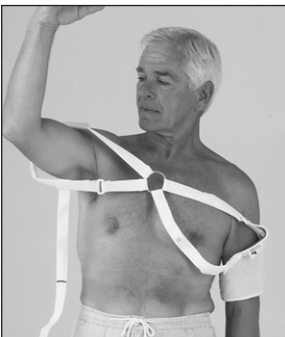
B

ENGLISH

Hold O-ring ring and pull strap assembly over head.

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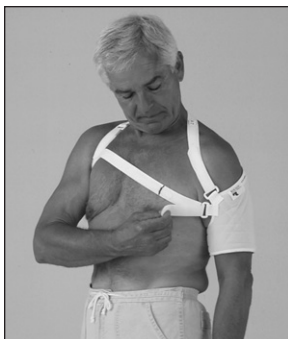
D

ENGLISH

Secure chest strap through D-ring to previously marked length.

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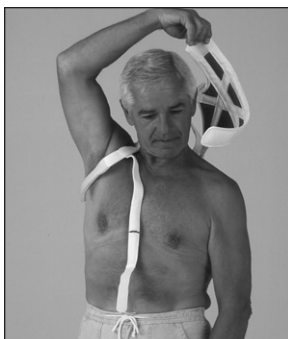
E

Unfasten chest strap and arm cuff. Bring arm cuff and strap assembly behind back and off unaffected arm

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