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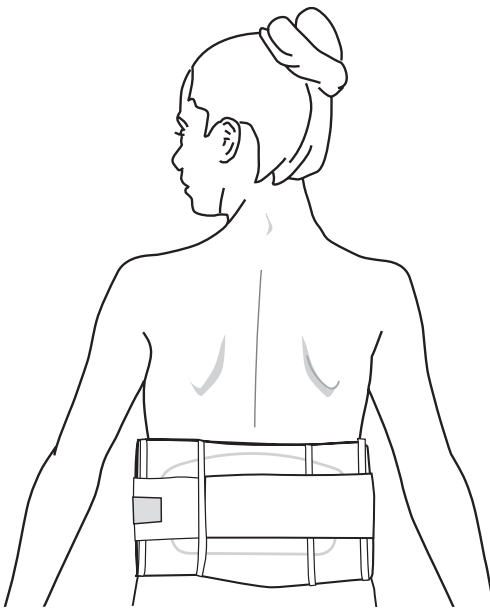
54 Hugh Adams Road • DeFuniak Springs, Florida USA 32435

Professional Products, Inc.

THERMAL FORM™ FOR LUMBOSACRAL SUPPORT

English

APPLICATION INSTRUCTIONS

INDICATIONS

For lumbosacral strains, sprains & fractures to the lower lumbar regions of the spine.
Also degenerative disk disease, aid to injury prevention & post-op support.

Thoroughly read the following information prior to application.
Product function requires proper application.

Unique features:

Material(s): Breathable Elastic Panels
Available with Flannel or Cotton-Lined Posterior Pocket for Thermal Form™ Insert
Breathable Elastic Tension Straps for Additional Support
Velcro® Closure

A

Place the insert Thermal Form™ Insert on a preheated hot plate (Fig. 1). The #1185-P1 Insert becomes flexible when warmed on a hot plate (Teflon® coated) for two minutes at 300°. The #1185-P2 Insert becomes flexible when warmed on a hot plate (Teflon® coated) for three minutes at 300°. Check flexibility of the insert periodically to insure against overheating and burning of insert.

B

The warmed insert is then taken off the hot plate and quickly placed in the pocket of the appropriate sized Lumbosacral Support (Fig. 2): P1 and P2 for #480, #1184, or #1083; the P1-X and P2-X for the #481.

C

The Lumbosacral Support is then quickly applied to the patient. (Fig. 3)

D

Secure with Velcro® closure (Fig. 4). With the patient in a standing position, the insert can now be molded to the patient's back for a custom fit.

CAUTION: The fitting and molding process must be done quickly as the moldable insert becomes cool and rigid in about three to five minutes after being removed from the hot plate.

USE GLOVES WHEN HANDLING THE HEATED INSERTS. DO NOT HEAT THE INSERT WITH AN AUTOCLAVE OR WITH A MICROWAVE OVEN.

CARE INSTRUCTIONS

Hand wash using cold water and mild detergent. Do not bleach. Rinse thoroughly, removing all soap residue to avoid skin irritation. Allow to AIR DRY. Do not twist or wring. reshape, dry flat

WARNING: THIS PRODUCT IS TO BE USED ONLY UNDER THE SUPERVISION OF A MEDICAL PROFESSIONAL. NOT INTENDED FOR UNSUPERVISED PUBLIC USE. INCIDENTS OF PAIN, SWELLING, CHANGES IN SENSATION OR UNUSUAL REACTIONS SHOULD BE IMMEDIATELY REPORTED TO YOUR MEDICAL PROFESSIONAL.

CAUTION: FEDERAL LAW (USA) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTHCARE PROFESSIONAL.

MANUFACTURER MAKES NO GUARANTEES THAT INJURY WILL BE PREVENTED USING THIS PRODUCT.

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