Español

INSTRUCCIONES DE APLICACION

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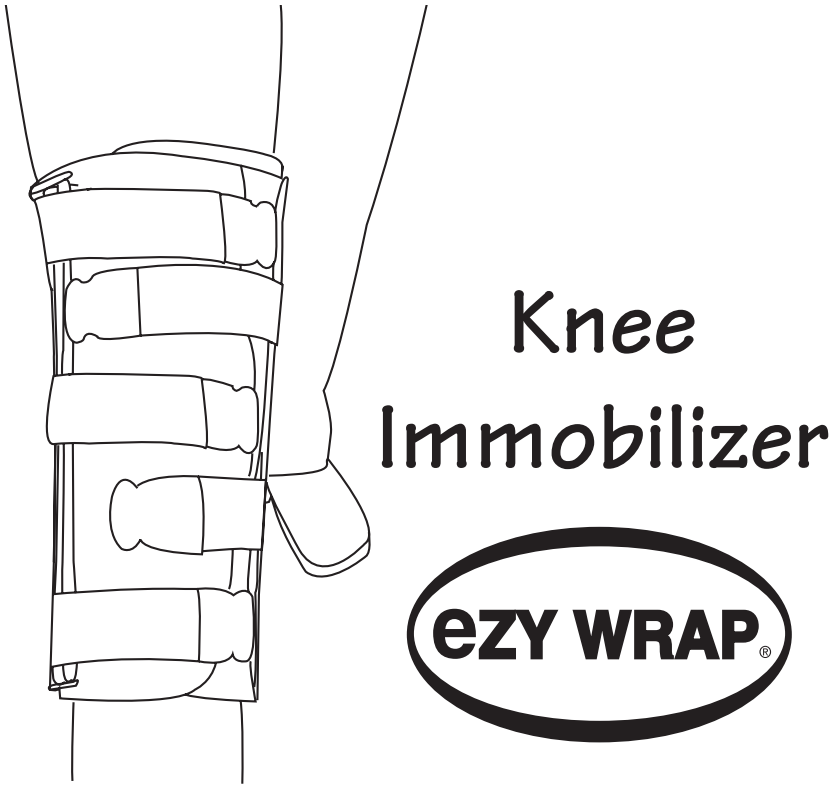
I / O
R000384

Copyright © 2001 Professional Products, Inc.

www.ezywrap.com

International (850) 892-5731 Fax (850) 892-5990
USA and Canada (800) 234-9004 Fax (800) 274-9006
54 Hugh Adams Road • DeFuniak Springs, Florida USA 32435

English

APPLICATION INSTRUCTIONS

INDICATIONS

Any pre- or post-operative condition when preventing flexion movement is beneficial. A temporary immobilization device while waiting for physician instruction in emergency situations.

Thoroughly read the following information prior to application.
Product function requires proper application.

Unique Features:

2" or 3" Elastic Compression Straps with contact closure
Velcro® hook "cheater straps"
Also available as a cold therapy product (See figure below and Section E)

Compression Straps

Wider End

Cheater Straps

Medial/Lateral

Knee Immobilizer

Gel Bags

Cloth Gel Bag Pocket(s)

Cold/Heat Therapy Knee Immobilizer

A Position correctly on leg:

Place immobilizer with posterior stays in the correct position on the leg, foam side to skin. Wider end of the immobilizer positioned toward the proximal position on the leg (Fig. 1).

B Close immobilizer:

Close immobilizer using "cheater" straps (Fig. 2).

C Place Medial/Lateral stays:

Place medial/lateral stays just posterior of the patella (Fig. 3 and Fig. 4).

D Secure Compression straps in proper sequence:

Pull compression straps around to fully enclose the leg and secure with Velcro® closure. Close straps in progression as shown in figure 5.

Note: The length of the immobilizer determines the number of straps on the brace.

E Cold/Heat Therapy Knee Immobilizer:

Insert frozen or heated gel bag into cloth pocket and secure using Velcro® closure (Fig. 6). Then follow the instructions as stated above.

Note : Please refer to instructions on bag for proper use of gel bag.

Fig. 1

Fig. 2

Fig. 3

Fig. 4

Fig. 5

Fig. 6

CARE INSTRUCTIONS

Hand wash using cold water and mild detergent. Do not bleach. Rinse thoroughly, removing all soap residue to avoid skin irritation. Allow to AIR DRY. Do not twist or wring. reshape, dry flat

WARNING: THIS PRODUCT IS TO BE USED ONLY UNDER THE SUPERVISION OF A MEDICAL PROFESSIONAL. NOT INTENDED FOR UNSUPERVISED PUBLIC USE. INCIDENTS OF PAIN, SWELLING, CHANGES IN SENSATION OR UNUSUAL REACTIONS SHOULD BE IMMEDIATELY REPORTED TO YOUR MEDICAL PROFESSIONAL.

CAUTION: FEDERAL LAW (USA) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTHCARE PROFESSIONAL.

MANUFACTURER MAKES NO GUARANTEES THAT INJURY WILL BE PREVENTED USING THIS PRODUCT.

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