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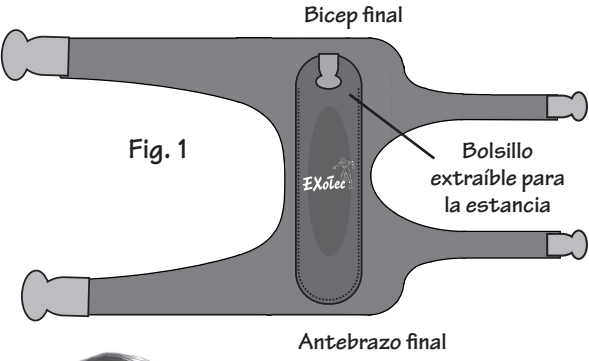
S / G
606000R

PROFESSIONAL PRODUCTS, INC.
54 HUGH ADAMS ROAD
DEFUNIAK SPRINGS, FL 32435
(800) 234-9004
(850) 892-5731
FAX (800) 274-9006

PROFESSIONAL PRODUCTS, INC.
54 HUGH ADAMS ROAD
DEFUNIAK SPRINGS, FL 32435
(800) 234-9004
(850) 892-5731
FAX (800) 274-9006

www.ezywrap.com

Cubital Tunnel Splint

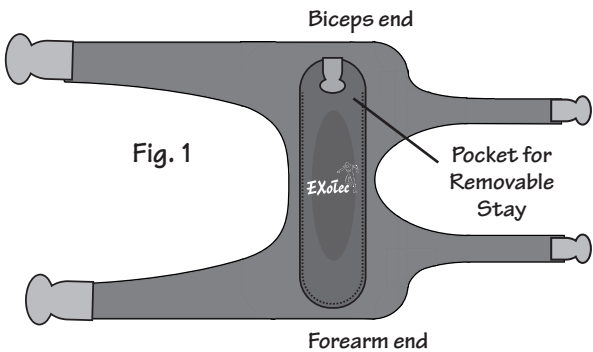
CE

EnglishAPPLICATION INSTRUCTIONS


INDICATIONS
Immobilizes the elbow to reduce pressure on the prominence of the ulnar nerve.
Restricts full extension and flexion to minimize stretching of the ulnar nerve.

Thoroughly read the following information prior to application.
Product function requires proper application.

A Loosen and unhook all straps.
(Fig. 1) Initially, splint is best applied with assistance.

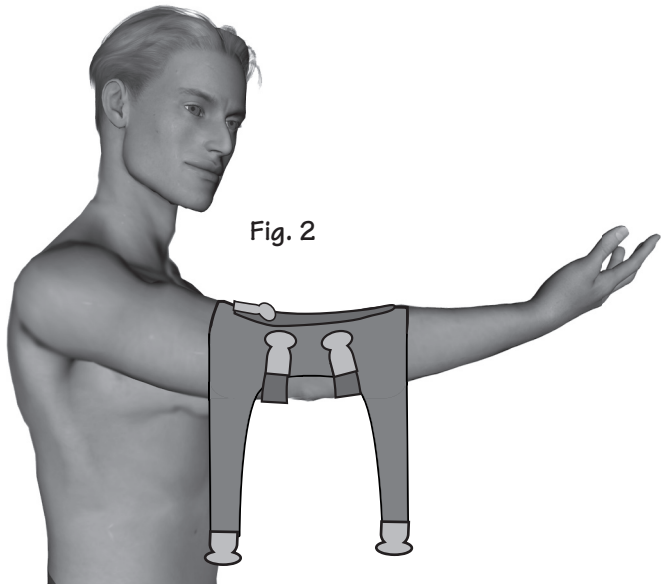B Place Splint on affected arm.
With arm slightly bent, position the logo directly over the elbow crease. (Fig. 2)


C Secure Splint.

- Secure middle straps as shown. (Fig. 2)
- Secure upper biceps and lower forearm straps. (Fig. 3)
- Do not over tighten.

Note:

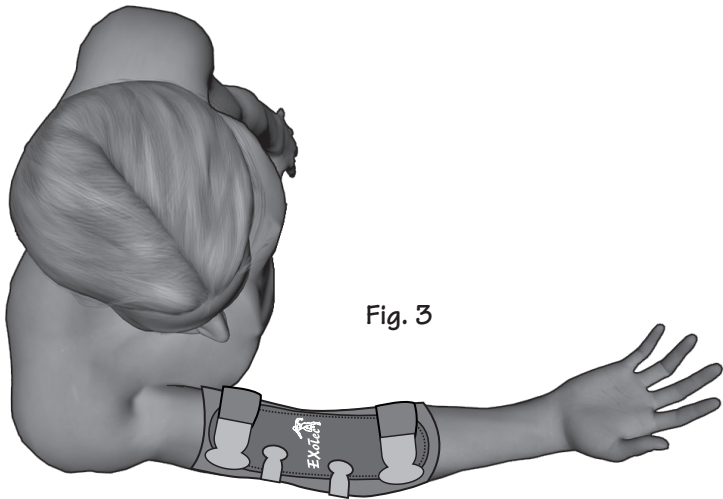- To facilitate re-application, when removing the splint, loosen but do not detach all straps then slide brace off.

CARE INSTRUCTIONS
Prior to cleaning, remove plastic stay from pocket. Hand wash using cold water and mild detergent. Do not bleach. Rinse thoroughly, removing all soap residue to avoid skin irritation. Allow to AIR DRY. Do not twist or wring. Reshape, dry flat. Wipe plastic stay clean with mild disinfectant. Replace plastic stay into pocket with textured surface facing outward, secure sleeve closure.

WARNING: INCIDENTS OF PAIN, SWELLING, CHANGES IN SENSATION OR UNUSUAL REACTIONS SHOULD BE IMMEDIATELY REPORTED TO YOUR MEDICAL PROFESSIONAL.
MANUFACTURER MAKES NO GUARANTEES THAT INJURY WILL BE PREVENTED USING THIS PRODUCT.

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