English



APPLICATION INSTRUCTIONS

INDICATIONS
Any condition that would require post-operative or post-trauma cold/heat therapy on or about the knee.

Thoroughly read the following information prior to application.
Product function requires proper application.



FIRSTICE® COLD THERAPY

- Apply optional FirstICE® Cold Therapy before applying wrap. For proper freezing, please refer to instructions on FirstICE® pack.
- Place FirstICE® packs in freezer for 4 to 12 hours prior to initial application.
 - To allow for proper freezing, do not stack.
 - Take FirstICE® packs out of freezer.
 - Gently massage packs to loosen granules.
 - Place frozen FirstICE® bags inside pocket and secure with Velcro® tab (Fig. 1).
 - Return used pack to freezer DFP.
 - Repeat this cycle as prescribed.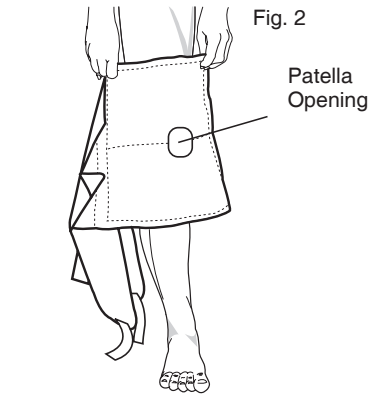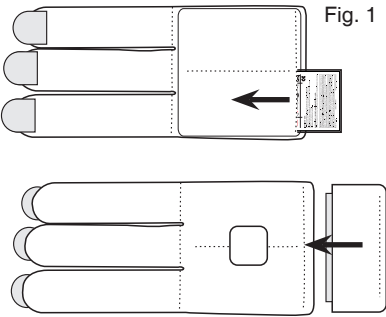

EZY-GEL™ COLD/THERMO THERAPY

- Apply optional EZY-GEL™ Cold/Thermo Therapy before applying wrap. For proper freezing/heating, please refer to instructions on gel pack.
- For first cold therapy use, place pack flat in freezer for at least 2 hours.
 - Remove from freezer or heat. Place gel packs inside pocket and secure with Velcro® tab (Fig. 1).
 - Leave in place for no more than 25 minutes or as recommended by physician.
 - Return used gel pack to freezer.
 - Repeat this cycle as prescribed.

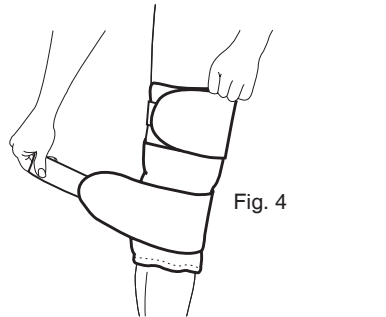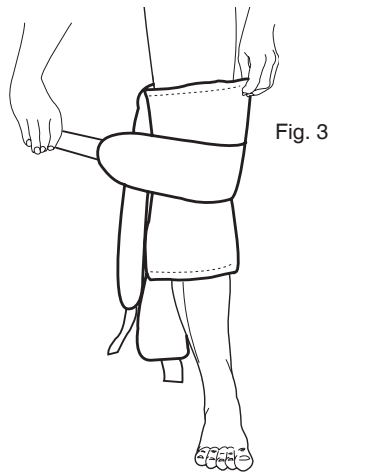KNEE WRAP

- A** Apply wrap:
With knee in slightly bent position, apply wrap with small circular cutout over the patella (Fig. 2). Add optional extender maxi pocket as needed for maximum circumferential cryotherapy (Fig.1)
- B** Secure straps:
Pull center strap around the leg and secure (Fig. 3). Repeat with strap above the patella; then secure strap below the patella (Fig. 4).

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