Español

INSTRUCCIONES DE APLICACION

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H / K
R000208

Copyright © 2001 Professional Products, Inc.

www.ezywrap.com


International (850) 892-5731 Fax (850) 892-5990

USA and Canada (800) 234-9004 Fax (800) 274-9006

54 Hugh Adams Road • DeFuniak Springs, Florida USA 32435

Post-Op Cold Therapy
Compression Shoulder Wrap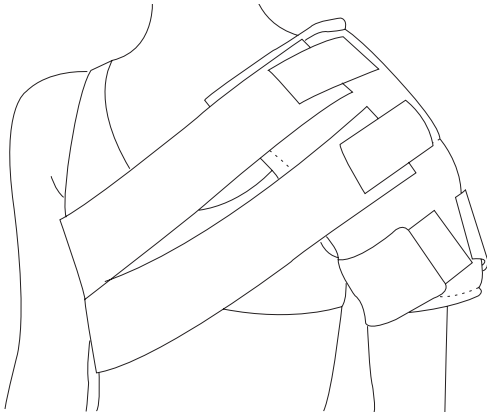

English

APPLICATION INSTRUCTIONS

INDICATIONS

For use after trauma injury, post-operatively or during rehabilitation to reduce pain and promote healing.

Thoroughly read the following information prior to application.
Product function requires proper application.

Unique features:


Y-body band specifically designed for women to provide a less restrictive fit.

Y-Body Band

Gel Bags

Shoulder Wrap

Arm Band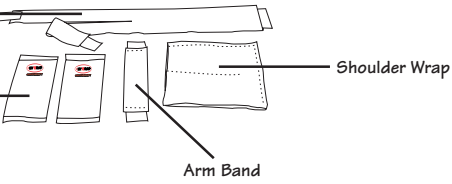

A

Insert and secure gel bags:

Place frozen or heated gel bags into cloth pockets and secure with Velcro® tabs (Fig. 1).

Note:

Please refer to instructions on bag for proper use of gel bags.

B

Attach and secure bands:

Turn over Shoulder Wrap and attach Y-Band and Arm Band. The straps should be attached at 45-degree angles at opposite corners (Fig. 2).

C

Apply Shoulder Wrap:

Place Shoulder Wrap on affected shoulder with the Y-Body Band to the back (Fig. 3).

D

Secure Arm Band:

Bring Arm Band around the under side of the arm and attach to the back of the Shoulder Wrap with the Velcro® tab (Fig. 4).

E

Secure Y-Body Band:

Bring band under the opposite arm, across the chest and secure the two ends of the Y section of the strap to the Shoulder Wrap with Velcro® tabs (Fig. 5).

F

Adjust Straps:

Adjust straps for optimal comfort. Open the Velcro® tab and trim the Y-Body Strap as needed. Reattach the Velcro® tabs and secure the strap ends to the Shoulder Wrap.


Fig. 1

Fig. 2

Fig. 3

Fig. 4

Fig. 5

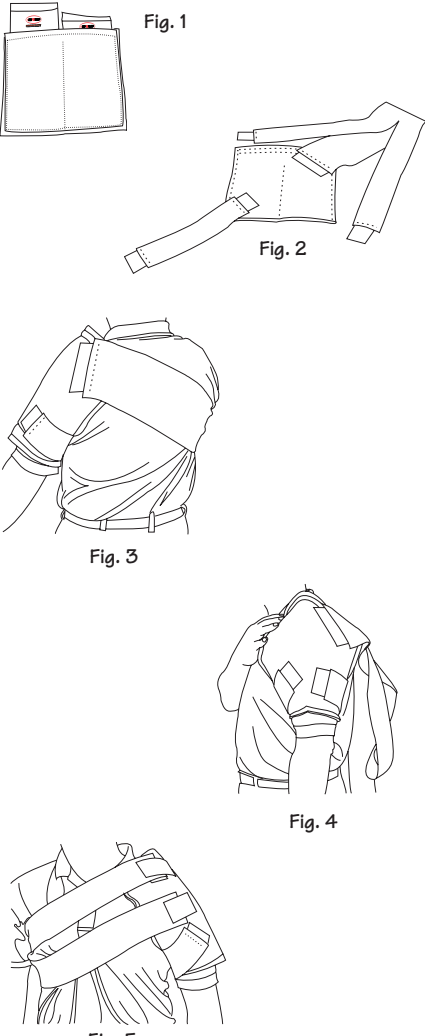CARE INSTRUCTIONS

Hand wash using cold water and a mild detergent. Rinse thoroughly and allow to air dry.

WARNING: THIS PRODUCT IS TO BE USED ONLY UNDER THE SUPERVISION OF A MEDICAL PROFESSIONAL. NOT INTENDED FOR UNSUPERVISED PUBLIC USE. INCIDENTS OF PAIN, SWELLING, CHANGES IN SENSATION OR UNUSUAL REACTIONS SHOULD BE IMMEDIATELY REPORTED TO YOUR MEDICAL PROFESSIONAL.

CAUTION: FEDERAL LAW (USA) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTHCARE PROFESSIONAL.

MANUFACTURER MAKES NO GUARANTEES THAT INJURY WILL BE PREVENTED USING THIS PRODUCT.

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