

APPLICATION INSTRUCTIONS

INDICATIONS

To provide support and control swelling after mild to severe ankle sprains.
Particularly suited for sports medicine applications.

Thoroughly read the following information prior to application.
Product function requires proper application.

HEAVYWEIGHT ELASTIC ANKLET



54 Hugh Adams Road • DeFuniak Springs, Florida USA 32435
USA and Canada (800) 234-9004 Fax (800) 274-9006
International (850) 892-5731 Fax (850) 892-5990

www.ezywrap.com

Copyright © 2007 Professional Products, Inc.

R000651
B / Q

Characteristics:
Heavyweight Breathable
Elastic that Contours to
Patient

Pullover Application

Open Heel

Fits Left and Right

A Pull on over affected
ankle. make sure no
wrinkles. Anklet should
fit tightly enough to
provide even, constant
pressure.

Fig. 1



Fig. 2



CARE INSTRUCTIONS

Hand wash using cold water and mild detergent. Do not bleach. Rinse thoroughly, removing all soap residue to avoid skin irritation. Allow to AIR DRY. Do not twist or wring, reshape, dry flat

WARNING: INCIDENTS OF PAIN, SWELLING, CHANGES IN SENSATION OR UNUSUAL REACTIONS SHOULD BE IMMEDIATELY REPORTED TO YOUR MEDICAL PROFESSIONAL.

MANUFACTURER MAKES NO GUARANTEES THAT INJURY WILL BE PREVENTED USING THIS PRODUCT.