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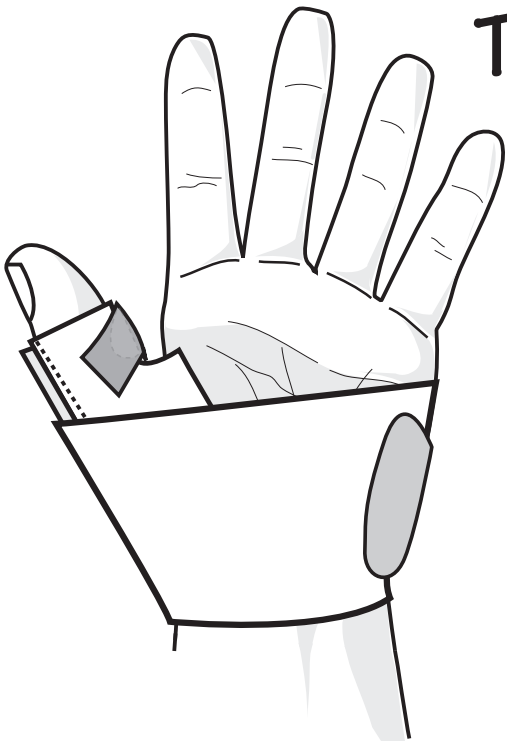
Copyright © 2005 Professional Products, Inc.

www.ezywrap.com
International
USA and Canada
(800) 234-9004
(850) 892-5731
DeFunia Springs, Florida USA 32435
54 Hugh Adams Drive

The Wahoo
Wrist
Support

INDICATIONS

Ideal for strains, sprains, the treatment of Carpal Tunnel Syndrome or thumb injuries.

Thoroughly read the following information prior to application.
Product function requires proper application.

Unique features: **Supreme Wahoo** features rigid aluminum thumb stay, removeable reinforcement thumb stay, and adjustable sized thumbhole.
Standard Wahoo features moldable aluminum thumb stay, contoured thumbhole, fits left (Fig. 3) or right (Fig. 4).

Supreme Wahoo

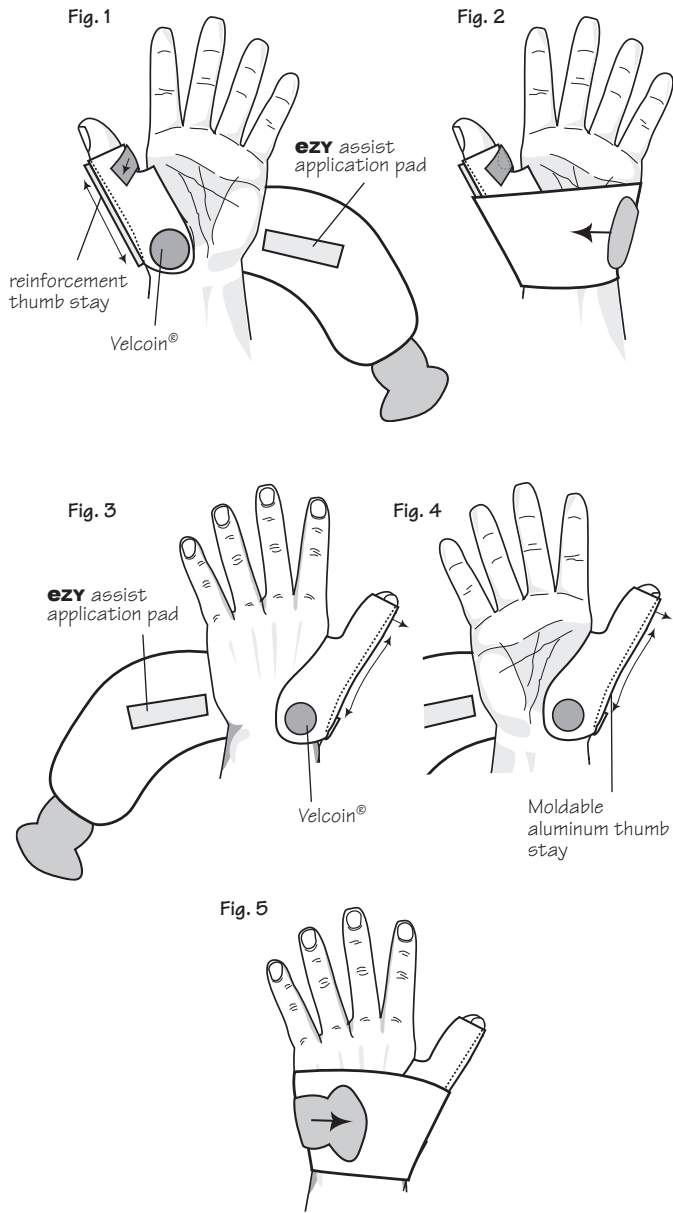
A Insert thumb into thumb hole.
Align reinforcement stay for optimal stabilization of thumb.
Close thumb strap around thumb with Velcro® tab. (Fig. 1)

B Complete wrapping.
Wrap compression strap around wrist. Secure **ezy** assist application pad to Velcoin®.
Adjust compression strap until it feels comfortable and secure (Fig. 2).

Standard Wahoo

A Insert thumb into contoured thumbhole. Contour shape moldable aluminum thumb stay as desired. (Fig. 3 Left), (Fig 4 Right)

B Complete wrapping.
Wrap compression strap around wrist. secure **ezy** assist application pad to Velcoin®.
Adjust compression strap until it feels comfortable and secure (Fig. 5).

CARE INSTRUCTIONS

Hand wash using cold water and mild detergent. Do not bleach. Rinse thoroughly, removing all soap residue to avoid skin irritation. Allow to AIR DRY. Do not twist or wring. reshape, dry flat

WARNING: THIS PRODUCT IS TO BE USED ONLY UNDER THE SUPERVISION OF A MEDICAL PROFESSIONAL. NOT INTENDED FOR UNSUPERVISED PUBLIC USE. INCIDENTS OF PAIN, SWELLING, CHANGES IN SENSATION OR UNUSUAL REACTIONS SHOULD BE IMMEDIATELY REPORTED TO YOUR MEDICAL PROFESSIONAL.

CAUTION: FEDERAL LAW (USA) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTHCARE PROFESSIONAL.

MANUFACTURER MAKES NO GUARANTEES THAT INJURY WILL BE PREVENTED USING THIS PRODUCT.

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