

APPLICATION INSTRUCTIONS

INDICATIONS

To provide support to abdominal area where muscles have been weakened due to surgery, pregnancy or abdominal hernias.

Thoroughly read the following information prior to application.
Product function requires proper application.

Fig. 1

A Correctly position binder:

Position binder around patient so Velcro® hook end is held in patient's right hand (Fig. 1.)



Fig. 2

B Wrap binder:

Wrap patient's left-hand side of binder over abdominal area first (Fig. 2).

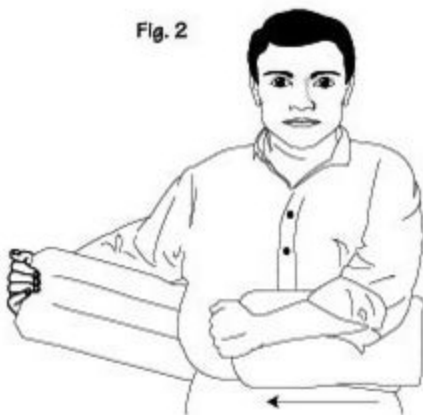


Fig. 3

C Wrap and secure:

Then wrap patient's right-hand side of binder over and secure it with Velcro® closure (Fig. 3).



CARE INSTRUCTIONS

Hand wash using cold water and mild detergent. Do not bleach. Rinse thoroughly, removing all soap residue to avoid skin irritation. Allow to AIR DRY. Do not twist or wring, reshape, dry flat

WARNING: THIS PRODUCT IS TO BE USED ONLY UNDER THE SUPERVISION OF A MEDICAL PROFESSIONAL. NOT INTENDED FOR UNSUPERVISED PUBLIC USE. INCIDENTS OF PAIN, SWELLING, CHANGES IN SENSATION OR UNUSUAL REACTIONS SHOULD BE IMMEDIATELY REPORTED TO YOUR MEDICAL PROFESSIONAL.

CAUTION: FEDERAL LAW (USA) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTHCARE PROFESSIONAL.

MANUFACTURER MAKES NO GUARANTEES THAT INJURY WILL BE PREVENTED USING THIS PRODUCT.