



Fitting instructions for supine patient



Step 1: Support patients head while sliding and positioning the back panel under the patients neck.



Step 2: Check positioning of back panel to make sure it is centered




Step 3: Place front chin piece underneath patients chin and on patients sternum.



Step 4: Unlock the dial by lifting up and rotating to raise the chin piece until it supports the patients chin. Make sure the patients chin sits behind the front edge of the collar.



Step 5: Once the correct height is achieved push down on the Trend dial to lock in place.

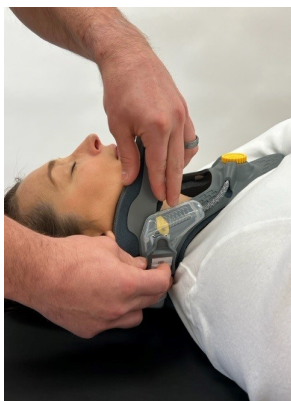

Step 6: Locate clips on left and right of the back panel and clip into front housing and pull until you hear a click



Step 7: To tighten detach the Velcro tabs and pull down to desired fit and reattach.



Step 8: Check fit and access patient.

Fitting Instructions for supine patient: Models DCT-0172, DCT-0174, and DCT-0180

Care Instructions: Padding can be removed from the plastic frame and hand washed with a light detergent. Replacement padding available at www.trend-med.com

Warranty: This product is covered by a 6 month warranty against all mechanical or manufacturer defects from the date of purchase of the original owner.

See reverse side for fitting instructions when patient is seated.

Fitting instruction for patient when seated



See reverse side for fitting instructions when patient is supine.



Step 1: Place collar under patient's chin.



Step 2: Locate the Trend Dial and pull up to unlock. Rotate the dial until the chin piece supports the patient's chin.



Step 3: Make sure patient's chin sits behind the front edge of the collar frame. Once the desired height is achieved push down on the Trend Dial to lock in place.



Step 4: Connect the back panel to the front by clipping the clips into front housing on both sides.



Step 5: Detach Velcro tabs and pull back to desired fit. Reattach tabs.



Step 6: Check fit on patient and note sizing. Review with patient how to reapply their collar.