FTC

DE Gebrauchsanweisung	3
EN Instructions for use	5
FR Instructions d'utilisation.....	7
ES Instrucciones de uso.....	9
PT (BR) Manual de utilização	11
NL Gebruiksaanwijzing	13

Fußkosmetik Montageanweisung

Product Number: FTC

Montage

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Cosmetic Foot Shell Instructions for Use

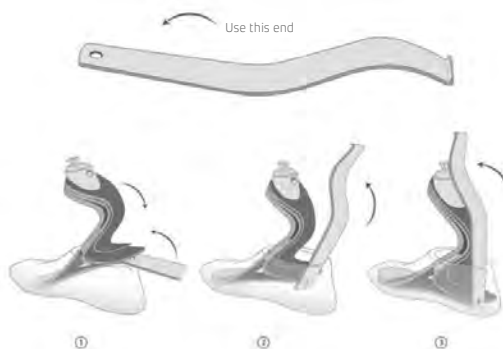
Product Number: FTC

Assembly


The Cosmetic Foot Shell is pre-assembled, consisting of the foot shell and the Cosmetic Attachment Plate (depending on version).

- Before installing the Cosmetic Foot Shell on the foot module, ensure the foot module is protected with one or more Spectra™ socks as appropriate.
- If equipped, remove the Cosmetic Attachment Plate.
- Insert the foot module into the Cosmetic Foot Shell using a Foot Shell Insertion/Removal Tool as shown in illustration below. On feet that use a soleplate (Renegade & Silhouette), it is advisable to first insert the soleplate into the shell and then the rest of the foot in a second step.
- Ensure the heel and/or sole plate of the foot module is correctly seated under the Cosmetic Foot Shell posterior heel tab.
- If equipped, attach the Cosmetic Attachment Plate to the Cosmetic Foot Shell.

Insertion Steps

Removal Steps

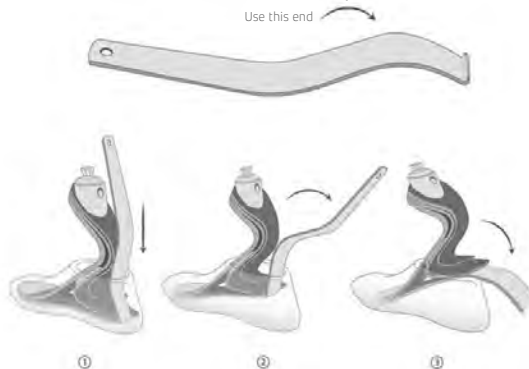Cosmetic Finishing

Use the provided Cosmetic Attachment Plate to attach the shin foam to the Cosmetic Foot Shell.

- Trace the shape of the Cosmetic Attachment plate to the shin foam.
- Contour the shin foam to match the shape of the Cosmetic Attachment Plate.
- Roughen the proximal bonding surface of the Cosmetic Attachment Plate.
- Adhere the Cosmetic Attachment Plate to the shin foam with contact adhesive and compress to ensure a secure bond.
- Orient the pins of the Cosmetic Attachment plate to the proximal pin holes of the Cosmetic Foot Shell. Ensure the pins are completely engaged.

Cosmetic Foot Shell

Available sizes: 16cm-31cm, foot module dependent

Heel height: foot dependent

Maintenance

The Cosmetic Foot Shell requires periodic maintenance.

- Inspect the Cosmetic Foot Shell every six months. If the user is more active, more frequent inspection may be necessary. Service as necessary. Replace the Spectra™ sock(s) and/or Cosmetic Foot Shell if worn to prevent damage to the graphite components.
- The Cosmetic Foot Shell may be cleaned and/or disinfected with mild soap and warm water.
- Do not allow aggregates such as sand to remain in the Cosmetic Foot Shell. Upon exposure to aggregates, immediately disassemble and rinse with water. The abrasive properties of aggregates will quickly wear the graphite components of the foot module.

Warnings

Failure to adhere to the guidelines of the Instructions for Use will void the warranty.

- Never use a foot module without a Cosmetic Foot Shell.
- Never use the Cosmetic Foot Shell without appropriate Spectra™ sock(s).
- Always wear the Cosmetic Foot Shell with footwear.
- Never use a screwdriver or any other instrument to remove the Cosmetic Foot Shell. Use only a Foot Shell Insertion/Removal Tool.
- The Cosmetic Foot Shell should only be used with Freedom Innovations foot products. Only use a Cosmetic Foot Shell with the same size (cm) as the foot module.
- The Cosmetic Foot Shell is not water resistant. Prolonged exposure to water may cause damage. Dry thoroughly before use if it becomes wet. If exposed to chlorinated/salt water, rinse with fresh water and dry thoroughly before using.
- The Cosmetic Foot Shell is not resistant to ultraviolet light (UV). Prolonged exposure to UV light may cause damage.
- The Cosmetic Foot Shell is not resistant to solvents. Exposure to solvents may cause damage.

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© Ottobock - 647G1733=all_INT-01-2102
Advena Limited. Tower Business Centre, 2nd Flr., Tower