

# Pediatric Foot Pyramid Adapter/Receiver Instructions

## WHAT'S IN THE BOX

One of the following:

Pediatric Titanium Foot Pyramid Adapter (PPF-1208)

Aluminum Foot Pyramid Receiver (PPF-1209)

Instructions

**Notice: WillowWood pediatric components have been tested and approved for use only with other WillowWood pediatric components. The use of WillowWood pediatric components with any manufacturer's adult-sized components will void the warranty. The use of WillowWood pediatric components with another manufacturer's pediatric components is done so at the sole judgment of the attending prosthetist.**

## RECOMMENDED INSTALLATION PROCEDURES

1. Place the Pediatric Foot Pyramid Adapter/Receiver onto the Pediatric SACH foot or Pediatric Impulse Foot so that the ridge on the distal surface of the Adapter/Receiver fits into the slot on the proximal surface of the foot.
2. Attach the Pediatric Foot Pyramid Adapter/Receiver using the foot bolt that was supplied with the foot. Apply Loctite 242 (or equivalent) to the foot bolt and tighten the bolt to 15 ft-lbs (20 Nm).
3. Install the mating pyramid component onto the Foot Pyramid Adapter/Receiver. If using the Foot Pyramid Receiver, apply Loctite 242 (or equivalent) to the setscrews and tighten to 3 ft-lbs (4 Nm).

**Note: Do not modify the Pediatric Foot Pyramid Adapter/Receiver in any way.**

## WARRANTY

The warranty for the Pediatric Foot Pyramid Adapters/Receivers is one year from date of invoice. The use of WillowWood pediatric components with any manufacturer's adult-sized components will void the warranty. Use of a Pediatric Foot Pyramid Adapter/Receiver for amputees whose adjusted body weight is greater than 132 lbs (60 kg) or who engage in extremely high and abusive activities is against WillowWood's recommendations and will void the one year warranty. Adjusted body weight is defined as the weight of the amputee plus any loads normally or routinely carried by the amputee. "Extremely high and abusive activities" are defined as activities such as skydiving, karate, and judo; activities that could result in injury to an individual's natural limbs; and activities that expose the prosthesis to corrosives such as salt water.

## WARRANTY DISCLAIMER

WillowWood warrants that each product manufactured will, at the time of delivery, be of workmanlike quality and substantially free of defects. WILLOWWOOD MAKES NO OTHER WARRANTY, IMPLIED, OR EXPRESSED, AND MAKES NO WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. This warranty shall terminate immediately upon an action to combine our products with other materials or in any manner to change the nature of our products. The sole remedy is replacement of the products or credit for the products. WillowWood's liability shall not exceed the purchase price of the product. WillowWood shall not be liable for any indirect, incidental, or consequential damage.

## WILLOWWOOD RETENTION OF RIGHTS

WillowWood retains all intellectual property rights reflected or incorporated in its physical products, regardless of the transfer of the physical products to another party or parties.

*WillowWood*<sup>®</sup>

The Ohio Willow Wood Company  
15441 Scioto Darby Road  
Mt. Sterling, OH 43143  
phone 740.869.3377 / 800.848.4930  
fax 740.869.4374 [www.willowwoodco.com](http://www.willowwoodco.com)

Ohio Willow Wood Company B.V.  
Keizersgracht 62/64  
1015 CS Amsterdam  
The Netherlands



PN-1671-E 11 JUN 2012