

PROFORMANCE[®]

MODIFIED ACRYLIC RESIN *STYRENE MONOMER INHIBITED*

DIRECTIONS: Use with Cream Hardener [CH-MAR] ONLY. Shake well before using. Weigh resin in mixing container, mix Cream Hardener into resin, and add 2% pigment (if applicable). Store away from heat.

GEL TIME [at 70°F]: 2% Cream Hardener (12 minutes); 3% Cream Hardener (10 minutes); 4% Cream Hardener (8 minutes). Times will vary based on room temperature.

H	2
F	3
R	2
PP	H

#PF-MARG, One Gallon

DISTRIBUTED BY
Cascade Orthopedic Supply, LP
800-888-0865
www.cascade-usa.com



WARNING:

FLAMMABLE. Use in a well-ventilated area away from heat or open flame.
Avoid contact with skin and eyes and prolonged inhalation of vapors.
In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.
KEEP OUT OF REACH OF CHILDREN.