



Parts Included

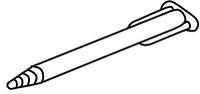

Small Mix Tip (1)

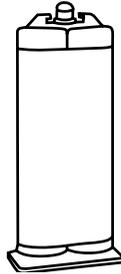

Coyote Quik Glue 50cc

Tech Tip -

Store all glue in cool environment for longevity.

Manufactured by



419 N. Curtis Rd., Boise, Idaho 83706
(208) 429-0026 | www.coyotedesign.com



QUIK GLUE CD4150

WHITE - 50cc - 30 second set time



Made in U.S.A.

External Prosthetic Components



EC REP




CD4150.revD.01142020




Advena Limited
Tower Business Centre
2nd Flr, Tower Street
Swatar, BKR 4013
Malta


Instructions

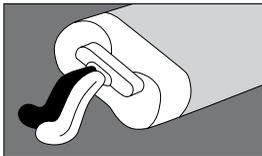


- 1** Purge the cartridge. Snip off the tip. Insert cartridge into dispenser gun. Expel a small amount of adhesive to purge the cartridge.

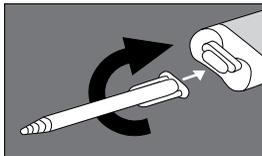

- 2** Install mix tip
Line up mix tip appropriately and twist into place.
Tips can only be used once.

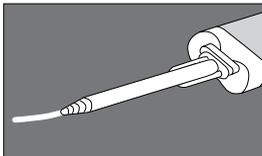

- 3** Run a test bead approx. 1" in length. Store unused portion by leaving the mix tip on when done.

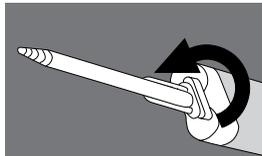


- 4** Leave used tip attached as a cap until next use to keep glue from drying out in the tubes.
When your ready to use the glue again remove old tip and replace with a new tip.

CAUTION

1. Read carefully BEFORE applying adhesive to ensure proper adhesion.
2. Quik Glue sets extremely fast. To ensure a proper bond, work very quickly. Refrigerate the cartridge to slow down the working time.

Parts Sold Separately

CDDMA51 | Dispenser is required for use (sold separately)

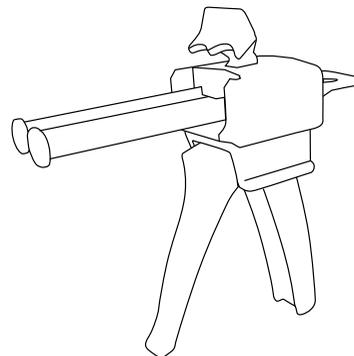

CD401 | 6-Pack Extra Mix Tips for 50 cc Cartridge.

CD402 | 12-Pack Extra Mix Tips for 50 cc Cartridge.



Contact Coyote for a MSDS for this product.
(208) 429-0026 or mailbox@coyotedesign.com