
Parts Included

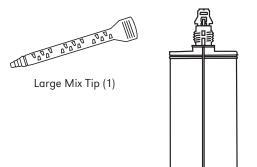

Coyote Quik Glue 220cc

Tech Tip -

Store all glue in cool environment for longevity.

Manufactured by



419 N. Curtis Rd., Boise, Idaho 83706 (208) 429-0026 | www.coyotedesign.com



QUIK GLUE

TAN - CD4320

60 second set time - 220cc




Made in U.S.A. External Prosthetic Components






Advena Limited Tower Business Centre 2nd Flr, Tower Street Swatar, BKR 4013 Malta


Instructions

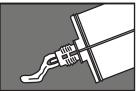


Purge the cartridge. Snap off the tip with screw driver. Insert cartridge into dispenser gun. Expel a small amount of adhesive to purge the cartridge.

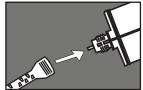

Install mix tip Line up mix tip appropriately and screw into place. Tips can only be used once.

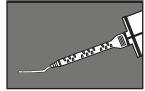

Run a test bead approx. 1" in length. Store unused portion by leaving the mix tip on when done.

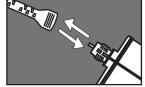

Leave used tip attached as a cap until next use to keep glue from drying out in the tubes. When your ready to use the glue again remove old tip and replace with a new tip.




CAUTION

- 1. Read carefully BEFORE applying adhesive to ensure proper adhesion.
- 2. Quik Glue sets fast. To ensure a proper bond, work quickly. Refrigerate the cartridge to slow down work time.

Parts Sold Separately

CDDM220 | Dispenser is required for use (sold separately)

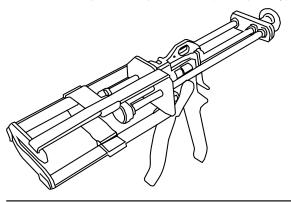

CD412 | 12-Pack Extra Mix Tips for 220 cc Cartridge.



Contact Coyote for a MSDS for this product. (208) 429-0026 or mailbox@coyotedesign.com