
Parts Included

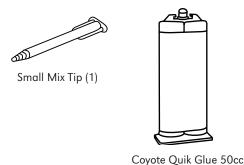

Tech Tip - Store all glue in cool environment for longevity.

Manufactured by



419 N. Curtis Rd., Boise, Idaho 83706 (208) 429-0026 | www.coyotedesign.com




QUIK GLUE TAN - CD4350

60 second set time - 50cc



Made in U.S.A.
External Prosthetic Components

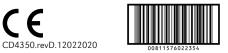



Advena Limited Tower Business Centre 2nd Flr, Tower Street Swatar, BKR 4013 Malta


Instructions




Purge the cartridge
Snip off the tip. Insert
cartridge into dispenser
gun. Expel a small amount
of adhesive to purge the
cartridge.



Run a test bead approx.
1" in length. Store unused portion by leaving the mix tip on when done.

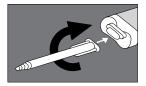

Install mix tip
Line up mix tip appropriately
and twist into place.
Tips can only be used
once.

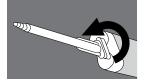

Leave used tip attached as a cap until next use to keep glue from drying out in the tubes.

When your ready to use the glue again remove old tip and replace with a new tip.




CAUTION

- 1. Read carefully BEFORE applying adhesive to ensure proper adhesion.
- 2. Quik Glue sets extremely fast. To ensure a proper bond, work very quickly. Refrigerate the cartridge to slow down the working time.

Parts Sold Separately

CDDMA51 | Dispenser is required for use (sold separately)

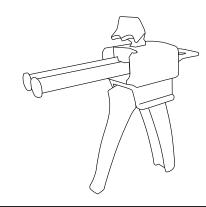

CD401 | 6-Pack Extra Mix Tips for 50 cc Cartridge.

CD402 | 12-Pack Extra Mix Tips for 50 cc Cartridge.



Contact Coyote for a MSDS for this product. (208) 429-0026 or mailbox@coyotedesign.com