



Seattle Triumph

- Narrow shank provides multi-axial compliance to uneven terrain increasing stability and limb comfort.
- Optional toe wedges provide increased dorsiflexion resistance and improved proprioception at mid-stance for increased stability.
- Multi-axial rotation and shock absorption.
- Full-length, split keel.
- Smooth rollover and controlled plantar flexion.
- Pre-assembled with Spectra sock in removable foot shell.
- Foot shell available in light and dark skin tones.

INDICATIONS

Medium to high impact, K3-K4, maximum weight of 160 kg • 350 lb

SELECTION


Model No.	Description	Product Size	Weight Limit
STF190	Seattle Triumph, Light	22-30 cm	160kg • 350 lb
STF193	Seattle Triumph, Dark	22-30 cm	160kg • 350 lb

All models come pre-assembled with foot shell and spectra sock.





Multi-axial rotation

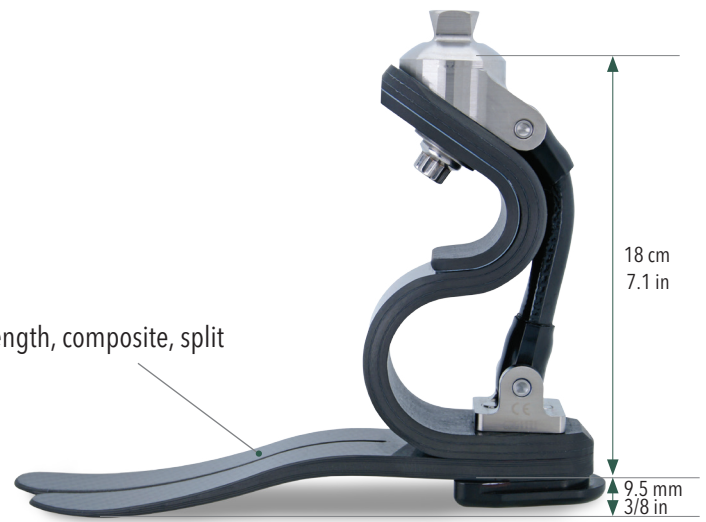

Full length, composite, split keel

Seattle Triumph

SPECIFICATIONS

Model No.	Weight Limit	Product Sizes	Product Weight	Heel Rise	Minimum Clearance	Warranty
STF 190, 193	160 kg · 350 lb	22-30 cm	493 g · 1 lb (size 25 without foot shell)	9.5 mm · 3/8 in	18 cm · 7.1 in	Foot: 3 years Foot Shell: 6 months

All models come pre-assembled with foot shell and Spectra sock.

CATEGORY SELECTION INSTRUCTIONS

To optimize the selection and ensure amputee's safety, follow the two steps below to determine the appropriate category.

1. Locate the column that corresponds with the amputee's impact level.
2. Within the selected column, locate the amputee's weight.

Important Note: If the amputee has a long BK, carries heavy loads or will reach a higher activity level within a year, choose the next category higher.

Warning: Choosing a lower strength category than what is suggested based on the above procedure and patient data will void the warranty. If your patients weight exceeds the limits of the chart please call Trulife Customer Service.

Category	Foot Length (cm)	Low Impact Level Walking Uneven Surfaces	Medium Impact Level Light Sports	High Impact Level Running, Basketball
9	26-30	137-160 kg · 301-350 lb	124-145 kg · 271-320 lb	106-125 kg · 232-274 lb
8	24-30	101-136 kg · 221-300 lb	89-123 kg · 194-270 lb	76-105 kg · 166-231 lb
6	23-29	81-100 kg · 177-220 lb	71-88 kg · 155-193 lb	61-75 kg · 133-165 lb
4	22-28	<80 kg · <176 lb	<70 kg · <154 lb	<60 kg · <132 lb

ACCESSORIES AND REPLACEMENT PARTS

Model No.	Description
SFC290	Foot Shell, Light
SFC293	Foot Shell, Dark

ORDERING EXAMPLE

Model No.	Size	Side	Category	Size	Category*	Last Digit of Size
STF190	28	L	6	L		6L8