

**A**

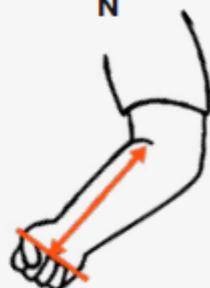
Measure the circumference of the widest part of the limb

**B**

Measure the circumference of the narrowest part of the limb

**C**

Measure the length from the crook of the elbow to the end of the limb

**N**

Measure the length of the opposite arm from the crook of the elbow to the middle of the palm

## Blank Sizing Chart

Use **measurement A** to select the appropriate size

### Measurement A

14cm – 18cm

### Blank size

Small blank

19cm – 24cm

Medium blank

25cm – 32cm

Large blank

33cm – 41cm

X-Large blank



Scan for our online sizing calculator

Enter the user's limb measurements, and the calculator will determine the correct ALX blank and dock size.

## Dock Sizing Chart

If fitting as **full** length use **measurement N – 1cm** to determine dock size for the chosen blank

If fitting as **limb** length use **measurement C + 7cm** to determine dock size for the chosen blank

### Blank Size

### Small Dock

### Medium Dock

### Large Dock

Small Blank

11cm – 31cm

Medium Blank

25cm – 37cm

17cm – 25cm

11cm – 17cm

Large Blank

29cm – 37cm

11cm – 29cm

X-Large Blank

17cm – 37cm