

When printing, please make sure your printer settings are set to print "Actual Size" to ensure the ruler on the sizing guide is accurate.

Aspen® Cervical Collar Sizing Guide


CHECK FOR A GOOD FIT

1. The chin must be flush with the front of the chin piece plastic.
2. The FlexTabs™ on the front and back panels should be flexed.
3. The collar should be "snug" but not touch the tracheal area.

PICK THE RIGHT SIZE

With the head in the neutral or desired position, place the guide on the highest point of the trapezius and flat against the side of the head.

Sight an imaginary perpendicular line from the bottom of the chin back to the guide and select the proper size Aspen Collar.

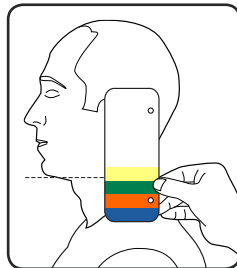

EXTRA TALL

TALL

REGULAR

SHORT

*Proper training in the use of this device should take place before it is applied.
These directions are guidelines only and are not offered as medical recommendations.*

Aspen® Pediatric Line Sizing Guide

A PROPER FIT AS EASY AS 1-2-3

1. Chin-Dots hidden?

The chin must be flush with the front of the chin piece plastic.

2. Hook and Loop past mark?

Back panel plastic must overlap the sides of the front panel.

3. FlexTabs™ bent?

The FlexTabs on the front and back panels should be flexed.

SIZING GUIDELINES

PD 1		1 - 18 months	weight 11 - 29 lbs (5 - 13 kg)
		length 21 - 33 in (53 - 84 cm)	
PD 2		9 - 24 months	weight 22 - 33 lbs (10 - 15 kg)
		length 29 - 37 in (74 - 94 cm)	
PD 3		1 - 3 years	weight 24 - 36 lbs (11 - 16 kg)
		length 33 - 40 in (84 - 102 cm)	
PD 4		2 - 5 years	weight 26 - 42 lbs (12 - 19 kg)
		length 35 - 45 in (89 - 114 cm)	
PD 5		3 - 6 years	weight 27 - 54 lbs (12 - 25 kg)
		length 37 - 48 in (94 - 122 cm)	

Aspen
MEDICAL PRODUCTS

Life Changing Innovation

Please Refer to Pediatric Collar Instruction Sheet for More Information.

© 2019 Aspen Medical Products. aspenmp.com
800-295-2776 • 949-681-0200 LIT5018E 08/19