English

PRODUCT LISTING

SKU	SIZE	SIDE
PA-ANS-SM	SMALL / MEDIUM	UNIVERSAL
PA-ANS-LXL	LARGE / X-LARGE	UNIVERSAL

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800-888-0865 | www.cascade-usa.com

PRODUCT DESCRIPTION

The ProActive Wear® Anterior Night Splint is low-profile and provides gentle stretching of the plantar fascia and Achilles tendon to help alleviate pain.

INDICATIONS FOR USE

- Plantar Fasciitis
- Achilles Tendon Repair

INSTRUCTION FOR PROPER USE

1. Refer to sizing chart to select proper size.
2. Detach all the padding straps.
3. Place foot into the brace with the knee flexed to achieve the maximum degree of dorsiflexion.
4. Secure the forefoot strap snugly by hook closure. Be careful not to over-tighten the strap.
5. Position the shell of the splint against the shin and wrap the calf strap around the lower leg to secure the brace.
6. Fasten the remaining heel straps snugly and locate the heel into the heel opening. Do not over-tighten the heel straps.

MAINTENANCE/CLEANING INSTRUCTIONS

Remove straps, liners, or pads off the plastic shell and close all hook and loop closures before washing. Hand wash liners with mild detergent in cold water and hang dry only. Do not dry clean, bleach, and iron.

CONTRAINDICATIONS

- Paraesthesia
- Severe varicose vein
- Skin disorders or injuries in the relevant area
- Impaired lymphatic drainage

SINGLE PATIENT USE

Single Patient Use Only

WARNINGS AND PRECAUTIONS

If allergic skin rashes occur, please consult your physicians immediately. The patient should be instructed to inspect their foot and ankle. This product is not intended for prolonged walking or weight bearing. Severe plantar flexion contracture or spasticity is contraindicated. Do not use continuously for more than 24 hours.

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