


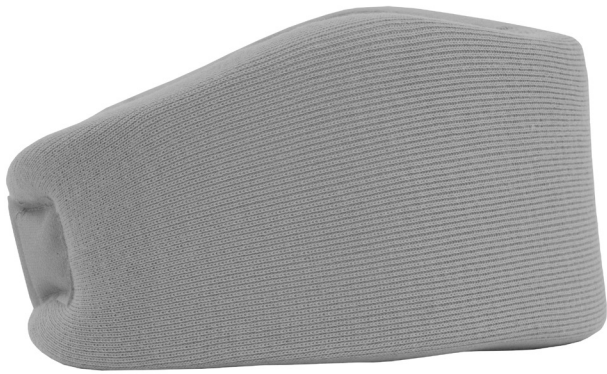

English

PRODUCT LISTING

SKU	SIZE
PA-CC-X-XS	X-SMALL
PA-CC-X-SM	SMALL
PA-CC-X-MD	MEDIUM
PA-CC-X-LG	LARGE
PA-CC-X-XL	X-LARGE

Français

LISTE DES PRODUITS

N° DE RÉF.	TAILLE
PA-CC-X-XS	T-PETIT
PA-CC-X-SM	PETIT
PA-CC-X-MD	MOYEN
PA-CC-X-LG	GRAND
PA-CC-X-XL	T-GRAND

PROACTIVE WEAR[®]

Padded Cervical Collar

Collet cervical en mousse

Instruction for Use
Mode d'emploi



800-888-0865 | www.cascade-usa.com

CASCADE ORTHOPEDIC SUPPLY, LP
2638 Aztec Drive | Chico, CA 95928
800-888-0865 | www.cascade-usa.com

PRODUCT DESCRIPTION

The ProActive Wear® Padded Cervical Collar helps to limit the movement of the patient's neck while providing support and comfort.

INDICATIONS FOR USE

- Acute neck pain
- Minor muscle spasm associated with spondylosis
- Protection during halo application

INSTRUCTION FOR PROPER USE

1. Measure circumference of neck.
2. Refer to sizing chart to select proper size.
3. Put collar on your neck.
4. Fasten the linking tape on cervical spine area.
5. Adjust compatibility.

MAINTENANCE/CLEANING INSTRUCTIONS

Wash in cold water, gentle cycle, with a mild soap. Do not dry clean or tumble dry. Do not use bleach. Air dry without exposure to any heat source.

CONTRAINDICATIONS

- Paraesthesia
- Severe varicose vein
- Skin disorders or injuries in the relevant area
- Impaired lymphatic drainage

SINGLE PATIENT USE

Single Patient Use Only

WARNINGS AND PRECAUTIONS

Do not wear supports if long-term covering cause allergic reactions, please consult your physicians. Do not continuously use for more than 24 hours.

DESCRIPTION DU PRODUIT

Le collet cervical en mousse ProActive Wear^{MD} sert à limiter les mouvements du cou du patient tout en lui apportant soutien et confort.

INDICATIONS D'EMPLOI

- Cervicalgies aiguës
- Spasme musculaire mineur associé à une spondylose
- Protection pendant l'application du halo crânien

INSTRUCTIONS POUR UNE UTILISATION CORRECTE

1. Mesurez la circonférence du cou.
2. Consultez le tableau des tailles pour trouver la grandeur appropriée.
3. Appliquez le collet autour du cou.
4. Fixez la sangle autoagrippante sur la nuque.
5. Ajustez selon vos besoins.

CONSIGNES D'ENTRETIEN ET DE NETTOYAGE

Lavez à l'eau froide, sur cycle délicat et avec un détergent doux. Ne pas nettoyer à sec ni sécher par culbutage. Ne pas utiliser d'eau de Javel. Sécher dans un courant d'air, et éviter d'exposer cette orthèse à toute source de chaleur.

CONTRE-INDICATIONS

- Paresthésie
- Varices sévères
- Troubles ou blessures cutanées affectant la région concernée
- Drainage lymphatique déficient

USAGE CHEZ UN SEUL PATIENT

Usage chez un seul patient

MISES EN GARDE ET PRÉCAUTIONS

Évitez de porter cette orthèse si le port à long terme provoque des réactions allergiques, veuillez consulter votre médecin. Évitez de l'utiliser en continu pendant plus de 24 heures.