025829 Rev A

CASCADE ORTHOPEDIC SUPPLY, LP
2638 Aztec Drive | Chico, CA 95928
800-888-0865 | www.cascade-usa.com

English

PRODUCT LISTING

SKU	SIZE	SIDE
PA-PNS-SM	SMALL	UNIVERSAL
PA-PNS-MD	MEDIUM	UNIVERSAL
PA-PNS-LG	LARGE	UNIVERSAL

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800-888-0865 | www.cascade-usa.com

PRODUCT DESCRIPTION

The ProActive Wear® Posterior Night Splint with adjustable flexion straps provides gentle stretching of the plantar fascia and Achilles tendon.

INDICATIONS FOR USE

- Plantar Fasciitis
- Achilles Tendonitis
- Drop Foot
- Post-Static Dyskinesia

INSTRUCTION FOR PROPER USE

1. Undo all buckle closures and place foot in splint with knee in flexed position to achieve the desired degree of dorsiflexion.
2. Attach the ankle strap. Adjust the length of the contact closure strap until comfortable. Do not over tighten straps. If numbness, tingling or other discomfort is experienced, loosen straps until comfortable.
3. Attach the foot strap. Adjust the length until comfortable.
4. Extend the knee to check the tightness of the ankle and foot straps. Adjust as needed.
5. Dorsiflexion straps alongside the medial and lateral sides of the leg can be gradually adjusted to increase or decrease the foot angle. Incremental adjustments may be necessary if discomfort is experienced.
6. Do not walk while wearing the splint.

MAINTENANCE/CLEANING INSTRUCTIONS

Clean with damp cloth. Dry thoroughly before next use. Liner may be hand laundered with mild detergent in warm water and air dried. Do not machine dry.

SINGLE PATIENT USE

Single Patient Use Only

WARNINGS AND PRECAUTIONS

This product is to be used under the supervision of a medical professional. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately. **Warning:** Do not attempt to weight bear or walk in this product.

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