



CASCADE ORTHOPEDIC SUPPLY, LP  
 2638 Aztec Drive | Chico, CA 95928  
 800-888-0865 | www.cascade-usa.com

## English

### PRODUCT LISTING

SKU	LENGTH	SIDE
PA-TPIK-12	12"	UNIVERSAL
PA-TPIK-16	16"	UNIVERSAL
PA-TPIK-18	18"	UNIVERSAL
PA-TPIK-20	20"	UNIVERSAL
PA-TPIK-24	24"	UNIVERSAL

## Français

### LISTE DES PRODUITS

N° DE RÉF.	LONGUEUR	CÔTÉ
PA-TPIK-12	305 MM (12")	UNIVERSEL
PA-TPIK-16	406 MM (16")	UNIVERSEL
PA-TPIK-18	457 MM (18")	UNIVERSEL
PA-TPIK-20	508 MM (20")	UNIVERSEL
PA-TPIK-24	610 MM (24")	UNIVERSEL

# PROACTIVE WEAR<sup>®</sup>

## Foam Tri-Panel Knee Immobilizer

Attelle d'immobilisation du genou  
universelle en mousse et à trois panneaux

Instruction for Use  
Mode d'emploi



800-888-0865 | www.cascade-usa.com

**PRODUCT DESCRIPTION**

The ProActive Wear® Foam Tri-Panel Knee Immobilizer provides a precise fit and is fully adjustable to accommodate varying leg circumferences.

**INDICATIONS FOR USE**

- Immobilization of the knee joint

**INSTRUCTION FOR PROPER USE**

1. Center the posterior stays in the popliteal area.
2. Position side panels for desired circumference. Center the patella opening on the knee.
3. Feed straps through D-rings and secure. Straps should be snug but not constrict.

**MAINTENANCE/CLEANING INSTRUCTIONS**

Hand wash liner in warm water using mild soap, rinse thoroughly. Air Dry. **NOTE:** If not rinsed thoroughly, residual soap may cause irritation and deteriorate material. For Knee Immobilizers with removable stays, remove stays prior to washing.

**SINGLE PATIENT USE**

Single Patient Use Only

**DESCRIPTION DU PRODUIT**

L'attelle d'immobilisation du genou universelle en mousse et à trois panneaux ProActive Wear<sup>MD</sup> assure un ajustement précis et est entièrement réglable pour convenir à différentes circonférences de jambe.

**INDICATIONS D'EMPLOI**

- Immobilisation de l'articulation du genou

**INSTRUCTIONS POUR UNE UTILISATION CORRECTE**

1. Centrez les baleines postérieures se situant dans la région poplitée.
2. Positionnez les panneaux latéraux de manière à obtenir la circonférence voulue. Centrez l'ouverture rotulienne sur le genou du patient.
3. Enfilez les sangles dans les anneaux en D et sécurisez-les en place. Les sangles doivent être bien ajustées sans être trop serrées.

**CONSIGNES D'ENTRETIEN ET DE NETTOYAGE**

Lavez la doublure à la main dans de l'eau tiède avec un détergent doux, puis rincez à fond. Sécher à l'air. **Remarque :** Les résidus de savon peuvent provoquer une irritation de la peau et une dégradation des matériaux si le produit n'est pas bien rincé. Si l'attelle d'immobilisation du genou est munie de baleines amovibles, retirez-les avant de la nettoyer.

**USAGE CHEZ UN SEUL PATIENT**

Usage chez un seul patient