


PROACTIVE WEAR®

Suspension Sleeves

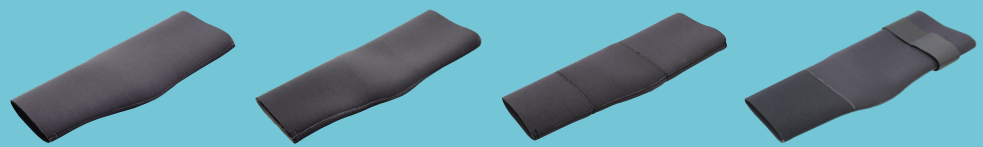

100 Series

200 Series

300 Series

400 Series

Color

Black

ProActive® Wear Knee Sleeves for auxiliary suspension provide outstanding suction with no cracking, fraying, or running. The sleeves are contour-shaped to the knee for patient comfort and reduction of bunching in the popliteal area. Each sleeve is bonded with liquid neoprene and blind stitched, creating a smooth seam to assure a comfortable fit. Four styles to choose from.

Sizing Information

Size	000	00	0	1	1.5	2	2.5	3	3.5	4	5
Circumference*	8–9.5	9–11	10.5–12	12.5–13	13.5–14	14.5–15	15.5–16	16.5–17	17.5–18	18.5–20	20.5–22

* Circumference measured in inches, 4" above knee center.

Donning Instructions

1. Fit knee sleeve over the prosthesis
2. Fold the top of the sleeve down
3. Position the residual limb in the socket
4. Grasp top edge of the sleeve and pull up gently
5. Position upper portion of sleeve on thigh for proper suspension and comfort

Care Instructions

Hand or machine wash gentle cycle for 5 to 10 minutes with warm water. Do not use bleach. Air dry.

Suspension Sleeve Materials LATEX FREE

100 Series

Exterior: Nylon

Interior: 3 mm Neoprene

200 Series

Exterior: Nylon

Interior: Nylon

300 Series

Exterior: Nylon

Interior:
Nylon Center;
3 mm Neoprene
Proximal and Distal

400 Series

Exterior:
Nylon with Strap

Interior:
Nylon Proximal and Center;
3 mm Neoprene Distal