


To determine the correct size, the measurement protocol starts by identifying the suspension type, followed by measuring the limb length and circumference at a specific height.


### STEP 1

which  
suspension ?

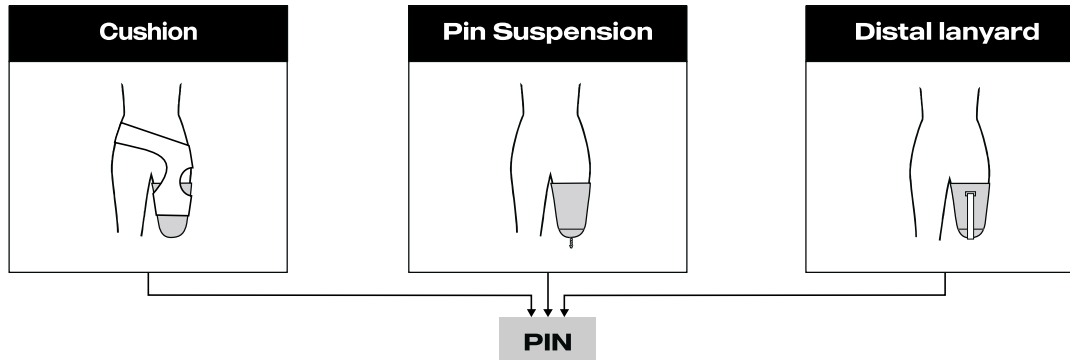


### STEP 2

which length ?

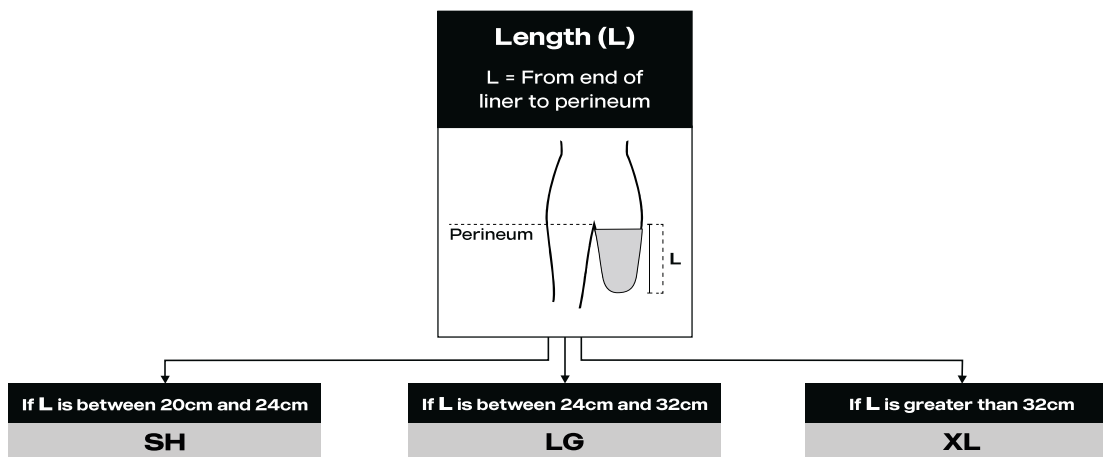

### STEP 3

which  
circumference ?

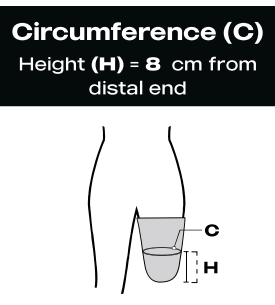

Refer to the chart for the product code  
based on measured circumference at  
**8 cm from distal end.**

Suspension	Length	Circumference	Code
Distal Locking + Lanyard +Cushion	20-24 cm	32-36 cm	OVTF-32-SH-PIN
		36-40 cm	OVTF-38-SH-PIN
		40-45 cm	OVTF-40-SH-PIN
		45-48 cm	OVTF-44-SH-PIN
		48-52 cm	OVTF-48-SH-PIN
		52-62 cm	OVTF-52-SH-PIN
	24-32 cm	32-36 cm	OVTF-32-LG-PIN
		36-40 cm	OVTF-38-LG-PIN
		40-45 cm	OVTF-40-LG-PIN
		45-48 cm	OVTF-44-LG-PIN
		48-52 cm	OVTF-48-LG-PIN
		52-62 cm	OVTF-52-LG-PIN

Suspension	Length	Circumference	Code
Distal Locking + Lanyard +Cushion	+32 cm	32-36 cm	OVTF-32-XL-PIN
		36-40 cm	OVTF-38-XL-PIN
		40-45 cm	OVTF-40-XL-PIN
		45-48 cm	OVTF-44-XL-PIN
		48-52 cm	OVTF-48-XL-PIN
		52-62 cm	OVTF-52-XL-PIN


Please see next page for  
**Vacuum suspensions** →

To determine the correct size, the measurement protocol starts by identifying the suspension type, followed by measuring the limb length and circumference on top of the seal.

### STEP 1

which suspension ?

#### Vacuum Suspension (Seal must be positioned distally)

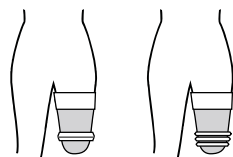


VAC

### STEP 2

which length ?

#### Length (L)

L = From top of seal to perineum

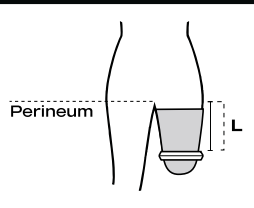

If L is between 20cm and 24cm

SH

If L is greater than 24cm


LG

### STEP 3

which circumference ?

#### Circumference (C)

Height (H) = 6cm on  
Top of seal

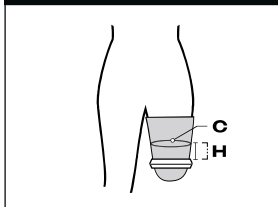

Refer to the chart for the product code based on measured circumference at **6 cm on top of the seal.**

Suspension	Length	Circumference	Code
Vacuum Suspension (Distally Positioned Seal)	20-24cm	32-38 cm	OVTF-32-SH-VAC
		38-42 cm	OVTF-38-SH-VAC
		42-47 cm	OVTF-40-SH-VAC
		47-50 cm	OVTF-44-SH-VAC
		50-54 cm	OVTF-48-SH-VAC
		54-62 cm	OVTF-52-SH-VAC
	+24 cm	32-38 cm	OVTF-32-LG-VAC
		38-42 cm	OVTF-38-LG-VAC
		42-47 cm	OVTF-40-LG-VAC
		47-50 cm	OVTF-44-LG-VAC
		50-54 cm	OVTF-48-LG-VAC
		54-62 cm	OVTF-52-LG-VAC

Please see next page for  
**Other suspensions** →