How to use

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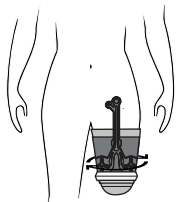
Distal integrated
Seal Only

Watch the video

Position the Overlay with the pump facing forward. (A slight rotation is possible if needed.)

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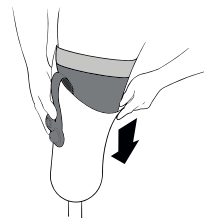
- ① Slide the Overlay over the liner and seal ring, with the air system on both sides of the limb**

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- ② Tuck the bottom of the Overlay as far as possible under the seal ring .**

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- ③ Slide the prosthesis back on, ensuring all air cells are inside the socket and below the trim line.**

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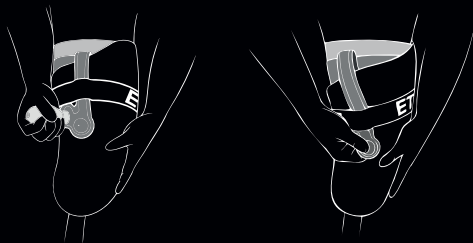
- ④ If desired, use the band to secure the pump and valve.**

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Inflate

Gonfler


- 1 **Adjust cushioning with the external pump until comfortable.**
- 2 **Use the integrated pump to fine-tune cushioning throughout the day.**

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Deflate

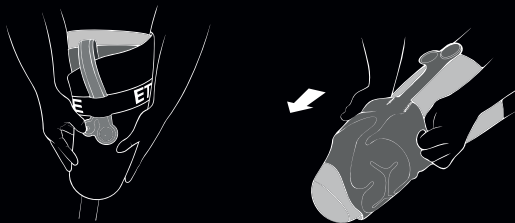Dégonfler

- 1 **Press the silver valve pin to deflate the Overlay.**
- 2 **Remove the prosthesis and the Overlay.**

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Once removed

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- 1 **Stretch and press the Overlay while pressing the exit valve to release remaining air.**

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Washing Precautions

- ⓘ Do not put in the washing machine and do not wring out the device to avoid damage.
 - ⓘ Do not expose the integrated pump and valve to water or fluid.
 - ⓘ Do not place it in a dryer or use any other heat source for drying.
 - ⓘ The use of unsuitable detergents may cause the device to deteriorate and void the warranty.
- Clean the device once a day only with approved cleansing products including neutral soap or mild detergent that is pH balanced

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