Elite Rehabilitator®
Fitting and Application

IMPORTANT

The Elite Rehabilitator® Ankle/Foot Orthosis (AFO) is designed to fit into most shoes without having to increase shoe size. It is important to wear a shoe with good support. Loafers, sandals, and shoes without a back should never be worn while using the Elite.

Each Elite Rehabilitator should be worn with a soft interface between the front of the brace and the leg. It is recommended to use the Elite Soft Kit for maximum comfort. Long, cotton socks may be used to increase comfort.

Applying Optional Soft Kit to your Brace

1. Remove one of the Soft Pads from the package along with the three (3) self-adhesive hook tabs.
2. Remove the backing from each of the three (3) hook tabs and apply to the inside of the brace cuff as shown.
3. Place the Soft Pad inside the cuff and press firmly so that the pad engages with the hook.

Fitting the Elite Rehabilitator

When possible, remove the inner sole of your shoe.

1. Place the brace into the shoe.
2. Replace inner sole of the shoe over footplate of the brace.
3. Slide your foot into the shoe. Fasten snugly (ties, Velcro, etc.)
4. Secure calf strap of your brace.


The cuff “wings” are flexible for your comfort, however excessive flexing of these wings back and forth can cause them to break.

Cleaning the Brace

Wipe brace with a damp cloth if needed.

It is recommended to wash your Soft Kit once a week.

1. Hand wash in cool water with gentle detergent. ***DO NOT USE CHLORINE BLEACH.***
2. Line drying is recommended, but a dryer on low heat and gentle cycle may be used.

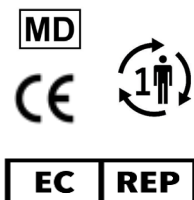Authorized Representative
MDSS GmbH, Schiffgraben 41
30175 Hanover, Germany

Elite Rehabilitator®
Fitting and Application

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GUARDIAN BRACE, Inc