

“Hammerhead” Kayak and Water Sports Prosthetic Accessory

The Hammerhead is specifically designed for kayaking but can be used with other rowing type water sports where the paddle/oar handles can be modified in diameter to meet Hammerhead requirements.

It is oriented on the prosthesis as illustrated in the picture below.

Place the Hammerhead’s concave surface against the kayak handle and tighten the strap around the paddle handle, securing it over the post retainer. IN Emergencies the strap can be instantly release by pulling on the Hammerhead’s “T” shaped handle releasing the paddle.

The wrist on your prosthesis should be adjusted to allow for the Hammerhead to have free rotation or rotation under friction, depending upon your preference. The Hammerhead can be used with disconnect wrists as well.

Store the Hammerhead with the strap coupling “Extended” (unattached) to the knob. This prevents the strap for developing a “set” and will provide for better long term performance.

Clean and wash the device regularly with dish detergent type soap and water, especially if used in salt water environments.

Do not leave the Hammerhead stored in direct sunlight for long periods of time. Most all polymers are impacted by ultra violet light exposure that slowly degrades the material properties.

Contact TRS at 1.800.279.1865 for more information, clarification, etc.

