Indications

> Chronic weak ankles, moderate to severe ankle sprains, immobilization following other ankle/foot injuries, preventative measure to assist in reducing frequency and severity of ankle injuries

Contraindications

> Fractures, open wounds

Cleaning Instructions/Maintenance

> The Mach8™ Ankle Brace is machine washable using a mild detergent and warm water.
 > To prevent the hook and loop straps from damaging other items in the wash, place the Mach8™ Ankle Brace in the included mesh bag before washing or wash separately.
 > Remove promptly and allow to air dry only.

Instructions for use

1. The Mach8 Ankle Brace is designed to be worn over athletic socks. Place your foot inside the Mach8 Ankle Brace and straighten and adjust the tongue for proper alignment. Ensure that all laces are on top of the tongue padding. Ensure your heel is positioned in the hole in the bottom rear of the brace. Feed the laces through the remaining eyelets and tighten to your preferred level of comfort and support. Tie the laces snugly.
2. Wrap the lateral (outside) strap across the top of the foot, under the heel, and back around to the lateral side. Pull up and secure the strap at your level of comfort and support with the hook & loop fastener on the lateral side of the brace.
3. Wrap the medial (inside) strap across the top of the foot and over the lateral strap, under the heel, and back around to the medial side. Pull up and secure the strap at your level of comfort and support with the hook & loop fastener on the medial side of the brace.
4. Grab one end of the top strap in each hand. One end of the top strap will have a red Thrive logo; this side is the "attaching end". One end of the top strap will not have a logo; this is the "receiving end".
5. Pull the "receiving end" strap (without the red Thrive logo) across the middle of the brace and over the exposed laces to your level of comfort and support.
6. Pull the "attaching end" strap (with the red Thrive logo) across the middle of the brace and over the "receiving end strap" to your level of comfort and support. Attach by firmly pressing the "attaching end" strap onto the "receiving end" strap. You are now ready to go!

Caution

Please always follow these instructions for use. In case of pain or swelling, discontinue use and consult your medical provider immediately. This device is designed to assist recovery and should be used exclusively under the instruction of and in consultation with a healthcare professional. Thrive Orthopedics does not assume any responsibility for misuse of the product. It is advisable to wear socks with the Mach8™ Ankle Brace. Do not drive a vehicle while you are wearing this device. Use extreme caution when walking on all surfaces when using this device. Walk more slowly than normal. This device is for single-patient use only.

To the patient/user: This device has been designed and manufactured by Thrive Orthopedics to be state of the art for treatment of a specific medical condition. Should any serious incident occur in relation to this device it should be reported to the manufacturer at the contact information listed on this document and the proper authority of the country or state where you are located. When treatment is complete, dispose of this device according to local laws and ordinances.

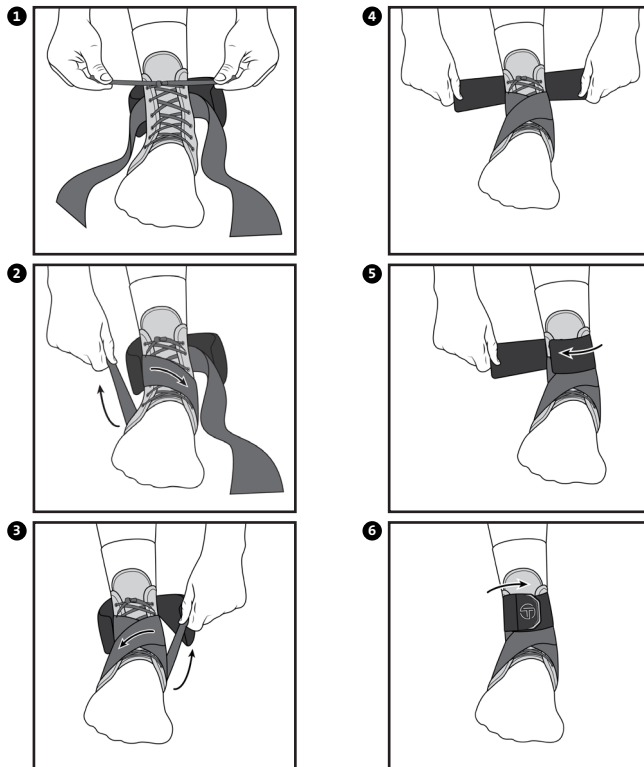
Manufactured for:
 Thrive Orthopedics LLC
 387 Ridge Point Drive
 Carmel, IN 46032
www.thriveorthopedics.com

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