Pipe with Adaptor N-P112

Instruction Manual

1. Information

This product must be assembled by a certified prosthetist. Please read this document carefully before using the product and observe the safety notices. Instruct the user in the safe use of the product.

Please contact the manufacturer if you have questions about the product or in case of problems.

Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.

Please keep this document for your records.

2. Purpose of use and product overview

This product is a pipe intended to be assembled with the intelligent knee joint (NI-C1 series, NI-C3 series, NI-C4 series) and a prosthetic foot via a male pyramid connector. It is for everyday life and can be used multiple times by one user.

You can adjust the angle of the sagittal plane and frontal plane of the prosthetic foot, length, internal rotation, and external rotation.

3. Product Specifications

- Maximum body weight: 100 kg (ISO 10328-P5-100kg)
- K Level (MOB): K2 Level (MOB2) ~ K4 Level (MOB4)
- Expected lifetime: 3 years
- Applicable environmental conditions
 - Storage temperature range: -20°C to +60°C (-4°F to +140°F)
 - Operating temperature range: -10°C to +40°C
 - Chemicals/liquids: Water or liquid (salt water, sweat, urine) splash
- Prohibited environmental conditions
 - Place of use: Underwater (in liquid)
 - Chemicals/liquids: Hydrochloric acid, sulfuric acid

4. Safety

Explanation of warning symbols

	Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.
	Indicates a potentially hazardous situation which, if not avoided, could result in minor or moderate injury.
	Indicates actions that you must not do when handling the product.

Strictly observe the following instructions to use the product safely.

- Tighten the set screws with the designated torque.
If the torque used to tighten the screws is less than the designated amount, the set screws could become loose and fall out during use.
- Adjust the alignment according to the instructions of the intelligent knee joint.

- DO NOT use the pipe with any other prosthetic knee than the intelligent knee joint.
- DO NOT use for a person who weighs over 100 kg (220 lb).
- When cutting the pipe, leave at least 90 mm from the end face on the adaptor side.
Failure to follow the instructions could cause the foot side to turn and thereby the user may fall.

- Do not leave or use the product in an environment not applicable.
- If there is any abnormality such as damage, abnormal noise, or backlash, discontinue use and take appropriate measures. (Contact the manufacturer or specialist workshop)
- Do not use the product beyond its useful lifetime.

5. How to use

You can use the pipe with adapter N-P112 by cutting it with its overall length left at least 90 mm.

* Cutting it to less than 90 mm will cause decline in the pipe clamp force of the intelligent knee joint.

- After cutting the pipe, remove burrs on both the inner side and outer side on the end face.
Failure to do so could cause you to cut your finger with burrs.


5.1. How to adjust the pipe length and the alignment of set screw

The length of the pipe and alignment must be adjusted by a prosthetist or a person who can handle prostheses.

- Mounting direction

Knee joint side: $\varnothing$ 34 pipe

Foot side: Female pyramid

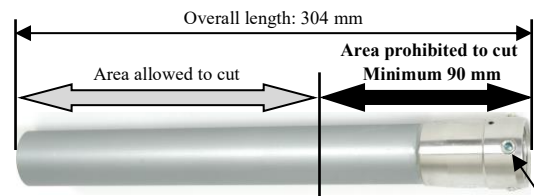Pyramid set screw (M8 x 12): 4 pieces
Tightening torque: 10N·m (102 kgf·cm)

* Apply the adhesive (equivalent to LOCTITE 243) on the thread part for anti-loosening of set screws, and tighten by a specified torque.

5.2. Cleaning

If necessary, wipe the surface with a soft cloth moistened with fresh water.

5.3. Maintenance

For safety, carry out regular inspections once a year for loose screws or abnormalities.

5.4. Disposal

This product is made of aluminum alloy and stainless alloy. Observe the information provided by the responsible authorities in your country regarding disposal procedures.

6. Warranty

- The warranty for the pipe with adaptor N-P112 is valid for a period of one year from the date of purchase.
- Note that if this product is failed or damaged due to improper use the warranty may become void even if it happens during the warranty period.
- Note that the warranty is not valid for failures and damage unless this product is used together with the intelligent knee joint.

7. Technical data

- Weight: 298 g
- Overall length: 304 mm
- Diameter of pipe: 34 mm

This device complies with the European Medical Device Regulation (EU) 2017/745.

Manufacturer

Nabtesco Corporation
Accessibility Innovations Company
Assistive Products Department
35, Uozakihamamachi, Higashinada-ku Kobe 658-0024, Japan
Phone: +81-78-413-2724 ; Fax: +81-78-413-2725
URL: <https://welfare.nabtesco.com>

Authorized representative for EU countries

PROTEOR SAS
6 rue de la Redoute
21850 Saint-Apollinaire, France
Phone: +33 3 80 78 42 42 Fax: +33 3 80 78 42 15
cs@proteor.com

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