

ASPEN® OTS

LUMBAR SACRAL ORTHOSIS
LUMBAL-SAKRALE ORTHESE
ORTESIS LUMBOSACRA
ORTHÈSE LOMBO-SACRÉE
LUMBALE SACRALE ORTHÈSE
BEĐERNÍ KRÍŽOVÁ ORTÉZA
LUMBAL SAKRAL ORTOSIS
ΟΣΦΥΚΗ ΟΡΘΩΣΗ ΙΕΡΟΥ ΟΣΤΕΟΥ
LUMBAL KORSRYGGORTROE

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





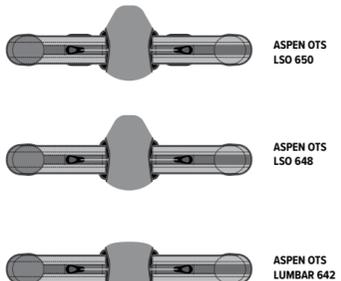


ASPEN OTS LSO 650
 ASPEN OTS LSO 648
 ASPEN OTS LUMBAR 642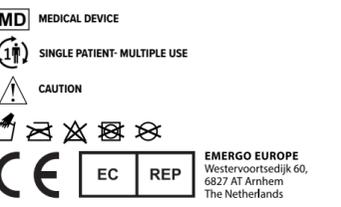

Pat. aspenmp.com/patents
 Aspen Medical Products, LLC
 6481 Oak Canyon, Irvine, CA 92618-5202 USA
 aspenmp.com • +1-800-295-2776 • +1-949-681-0200
 ©2023 Aspen Medical Products, LLC. All Rights Reserved.
 OS0002D revised 03/2023

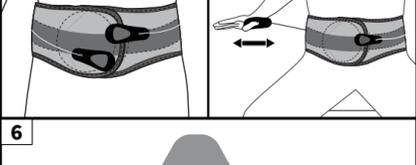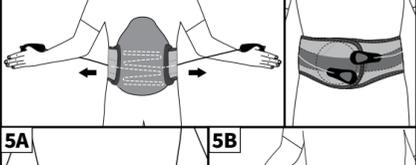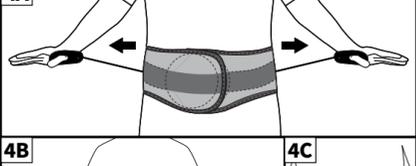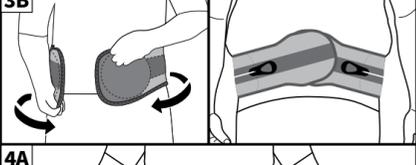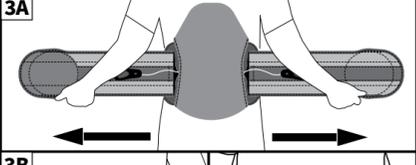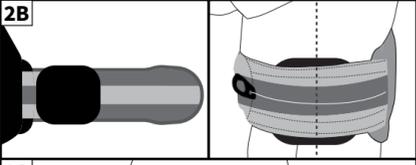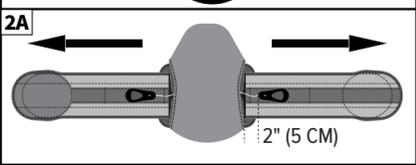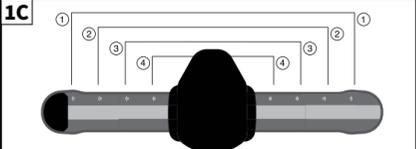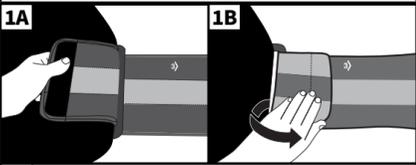ENGLISH
1. SIZE ADJUSTMENT
 A. To adjust the size of the brace, peel up the inside end of each side panel.
 B. Adjust as necessary to achieve a proper fit and reach.
 C. Brace sizes follow closely to standard pants sizes. Size ranges are only guidelines.
NOTE: Once the brace is applied, the sides of the brace may require further adjustment to achieve a snug fit without too much overlap at the front of the brace.

SIZING GUIDE		
Size	Circumference	
1	28 in	71 cm
2	37 in	94 cm
3	45 in	114 cm
4	54 in	137 cm

2. PREPARATION
 A. Place the pull tabs 2 inches (5 cm) from the plastic in the back of the brace. Pull the ends of the brace away from each other ensuring that the brace is fully extended.
ASPEN OTS LSO 650:
 B. With brace extended fully, place one lateral panel on midline on each side panel of the brace. Press down to secure to brace.

3. APPLICATION
 A. Apply the fully extended brace making sure the back panel is centered and low on the back.
 B. Apply the brace tightly, with the right side overlapping the left. When applied properly, both the upper and lower edge of the brace should be snug. If they are not, reapply the brace. If needed, angle the ends to insure a snug fit at both the upper and lower edges.
NOTE: For certain patients, the ends must be steeply angled.

4. TIGHTENING
 A. Slip your thumbs through the holes in the pull tabs. Pull away from your body.
 B. The left pull tab controls lower compression and the right pull tab controls the upper B. compression. Adjust the compression as needed.
 C. When tightened, secure the pull tabs to the loop in front of the brace.

5. ADJUSTMENT
 A. Adjust the upper and/or lower compression by loosening or tightening the pull tabs as needed.
 B. When patient is seated, slightly loosening the upper portion may be beneficial.

6. PANEL REMOVAL
 If necessary, remove back panel after disengaging hook and loop attachment between back panel and belt.

INTENDED USE
 To provide motion restriction and stability of the lumbar-sacral spine.

INDICATIONS
 Postoperative support, Deconditioned trunk musculature, Post laminectomy syndrome, Surgical Decompression procedures, Non-surgical Decompression procedures, Degenerative disc disease, Bulging or herniated disc, Non-specific low back pain, Chronic low back pain, Fracture management, Spondylolysis, Spondylolisthesis, Spinal stenosis, Ligament sprains/ Muscle strains, Spondylosis (Osteoarthritis/Degenerative Joint Disease), Facet syndrome, SI dysfunction, Sacroiliitis, Radiculopathy, Injections, Radiofrequency Ablation, Lumbar Multifidus Muscle Atrophy.

CONTRAINDICATIONS
 Hypersensitivity or allergies to any of the materials from which the brace is made. Swelling of the lymphatic tissue caused by circulatory disorders. Patients who are not allowed to wear a brace according to medical instructions.

PRECAUTIONS
READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations. If you suffer from a serious medical condition, we strongly suggest that you consult with a licensed health care professional before using this product. Proper fitting is required for this product to be effective. Under some circumstances, this product may be prescribed by a physician. Please see the limited warranty for further information.

CARE INSTRUCTIONS
 Hand wash only at 30° C. Wash with mild detergent; Air dry. Do not use bleach or other chemicals.

MATERIALS
 Nylon, Polyester, HDPE, PVC, Polyurethane, UHMWPE.
NOT MADE WITH NATURAL RUBBER LATEX.

DISPOSAL
 Product may be safely disposed in accordance with local laws.

REPORTING
 Please report any serious incident involving the use of this device to both the Competent Authority in your state and to the manufacturer (listed on this document).

LIMITED WARRANTY
 Aspen Medical Products, LLC, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products, LLC in the event of breach of warranty shall be to repair or replace the defective product or part(s).

Aspen Medical Products, LLC shall have no obligation under this limited warranty in the event:
 (a) The product was not purchased from Aspen Medical Products, LLC or through its authorized channels of distribution;
 (b) The product is altered;
 (c) Any parts not supplied by Aspen Medical Products, LLC are inserted into the product or
 (d) The product is not used in accordance with the Aspen Medical Products, LLC instructions for use.
THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF OR DEFECT IN THE PRODUCT. ASPEN MEDICAL PRODUCTS, LLC SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHETHER INCURRED UNDER COMMON LAW OR STATUTE.
 This warranty gives you specific legal rights and you may have other rights which vary from location to location.
 A listing of authorized distributors in your area is available upon request.

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