Overlay pump maintenance instructions

Proceed only if the Overlay is losing pressure


Instruction video

[Click here](#)

1 Read maintenance precautions below

- ⚠ Do not press the integrated pump or valve while the Overlay is submerged in water.
- ⚠ Only isopropyl alcohol should be used. At least 70% alcohol, also known as rubbing alcohol, available in most pharmacies.
- ⚠ When using alcohol, submerge only the tip of the integrated pump (the part with the hole).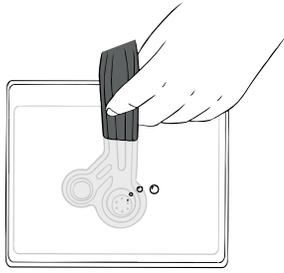


- 2 Inflate the Overlay fully. Then submerge the pump in water and observe carefully.

If no air bubbles come from the pump after 1 minute, or if bubbles appear elsewhere on the product, please contact us at: clinics@ethnocare.ca


- 3 Remove the Overlay from water and deflate it completely.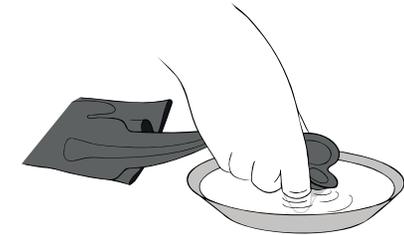


- 4 Submerge only the tip of the pump (the part with the hole) into isopropyl alcohol.


Then pump a small amount of alcohol into the Overlay.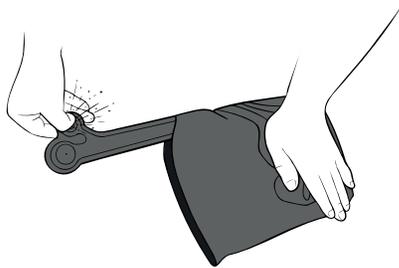

- 5 Release the excess alcohol by pressing on the valve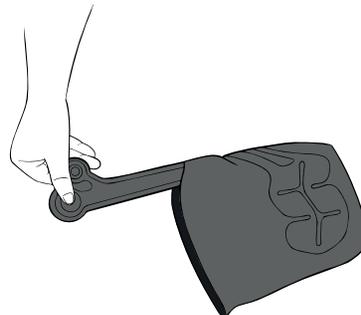

- 6 Inflate the Overlay using the integrated pump.

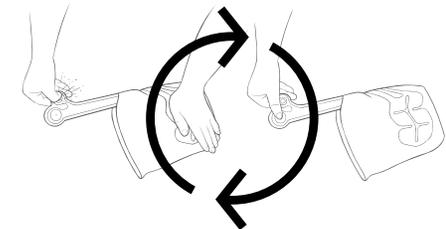- 7 Repeat steps 5 and 6 a few times to remove all remaining alcohol from inside the bladders.

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